




Assembling the 8-in-1 DermiSystem

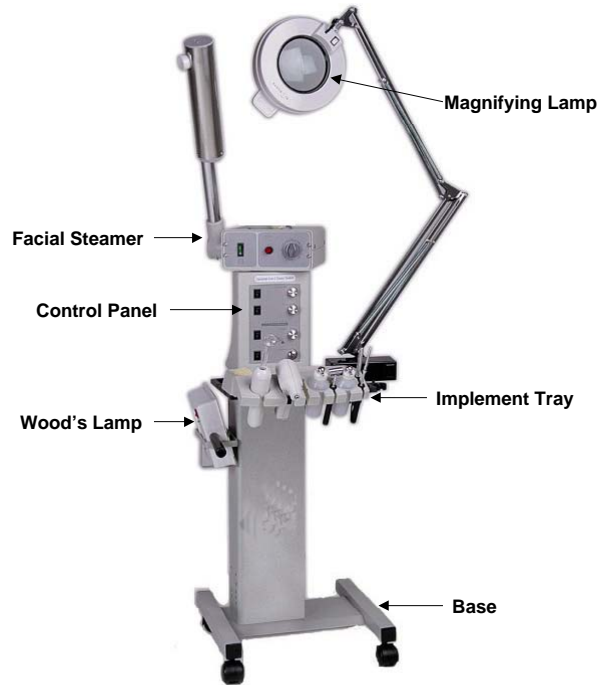

Note: Individual components are labeled at the appropriate places in the manual.

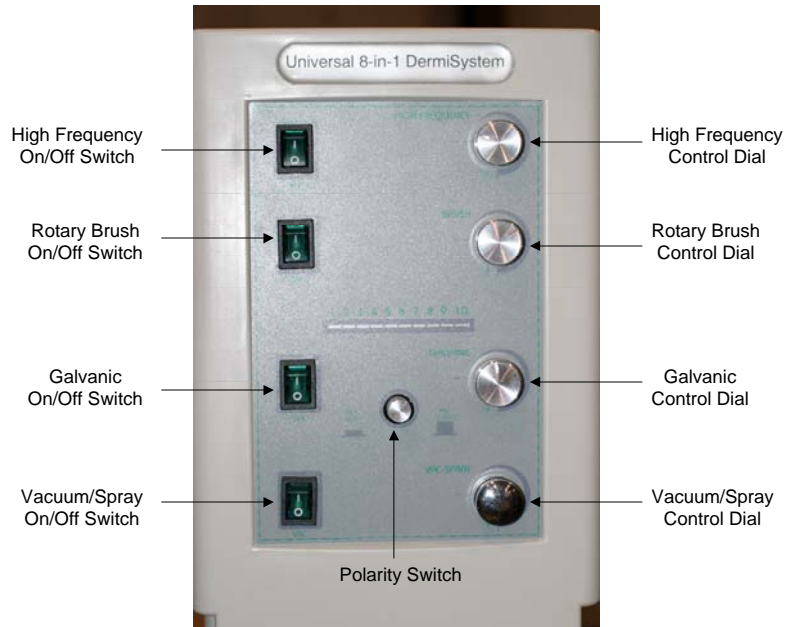

Control Panel

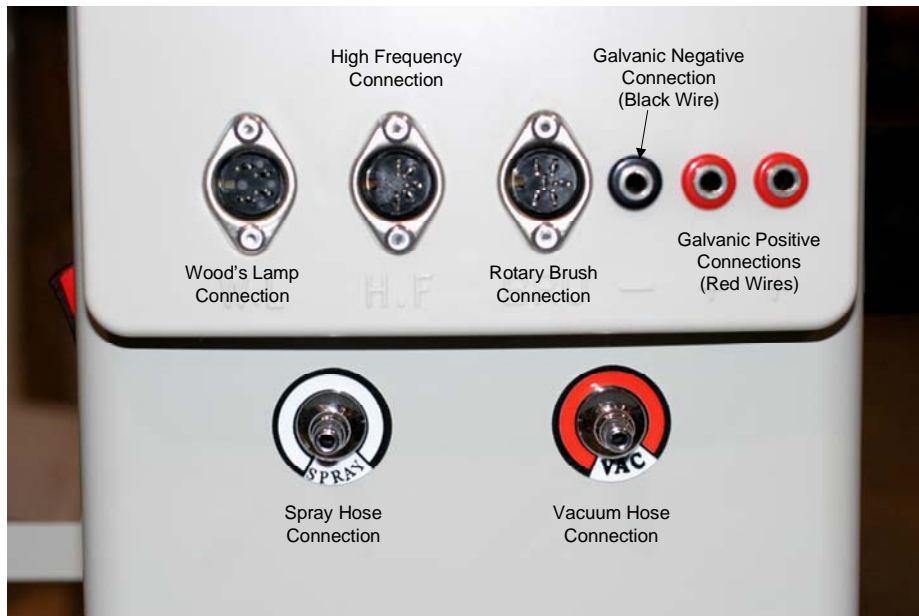

**Front View of the 8-in-1 DermiSystem
Plug-Ins for Various Functions
(located below the implement tray)**



Main Power Switch (located on left hand side of the base)



Outlets for Steamer and Mag Light (located on right hand side of the base)

There are no tools required to assemble the 8-in-1 DermiSystem. However, two people may be required to lift the main base unit from the box. It is heavy and slightly awkward for one person to handle. The 8-in-1 DermiSystem comes in two boxes.

1. Open the larger of the two boxes. Remove the top piece of foam. The box should contain:
 - ◆ 8-in-1 Base
 - ◆ Magnifying Lamp
 - ◆ Accessories
2. Remove the pieces of foam at the bottom of the unit. Their purpose is to support/protect the base/casters during shipping.



Note: Save the packing materials. If you ever need to ship this unit to another location, this piece of foam must be in place to support the unit. If it isn't in place and the box is dropped during shipping, the bottom casters could break off.

3. Lift the base out and set it upright on the floor. *Note: Two people may be required to safely perform this task.*
4. Remove the plastic cover and inspect the unit for shipping damage.
5. Remove the magnifying lamp from the box and insert it in the support bracket on the right hand side of the unit, making sure that the cord doesn't get bound up.
6. Remove the plastic covering and inspect the magnifying lamp for shipping damage.
7. Remove the wire tie that is holding the arm of the magnifying lamp in a stable position.



8. To adjust the position of the lamp, loosen (one at a time) the three knobs located at the base of the arm, the elbow, and the head. When you have adjusted the lamp to the desired position, tighten the knobs. Don't over tighten.

Note: The adjustment knobs must be loosened every time you want to adjust the position of the lamp. If you try to adjust its position without loosening the adjustment knobs, you may strip them, and the lamp will not stay in the desired position.




9. Plug the Power Cord for the Magnifying Lamp into one of the two outlets located on the side of the stand just below the support bracket. *(Note: The Steamer will be plugged into the other outlet.)*
10. The accessories are located in a clear plastic box, which you will find under the lower piece of styrofoam. It should contain:

- ◆ Wood's Lamp
- ◆ Cover for Wood's Lamp
- ◆ Vacuum/Spray Tubing
- ◆ 2 Spray Bottles
- ◆ 3 cupping glasses (straight, wrinkle, blackhead)
- ◆ 4 High Frequency Electrodes (acne, mushroom, indirect, spoon)
- ◆ High Frequency Handle
- ◆ Rotary Brush Handle
- ◆ 6 Rotary Brush Attachments (pumice, 2 foam, 3 goat hair brush)
- ◆ Power Cord
- ◆ 2 Red Galvanic wires
- ◆ 1 Black Galvanic Wire
- ◆ Galvanic Electrodes (2 roller, 2 pen, 1 fork or split, 1 plate, 1 ball)




Place the accessories in the appropriate slots on the Implement Tray.

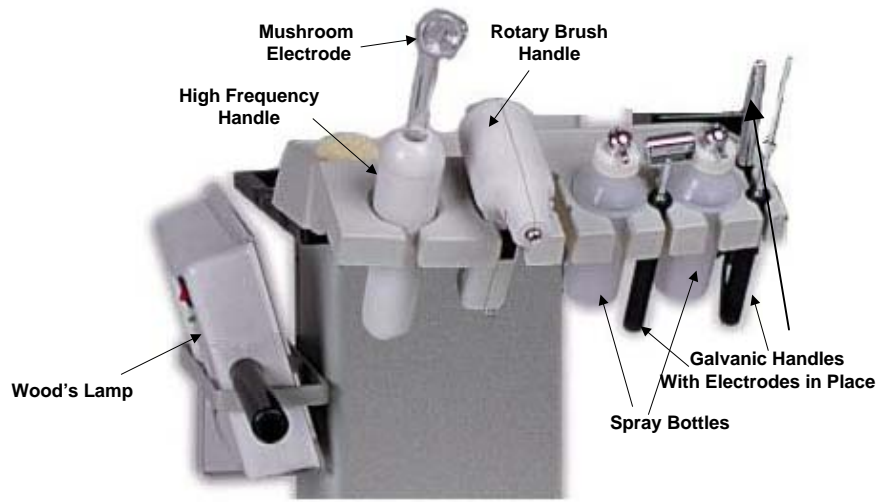

Implement Tray

11. Plug the various cords into the appropriate sockets (see diagram).

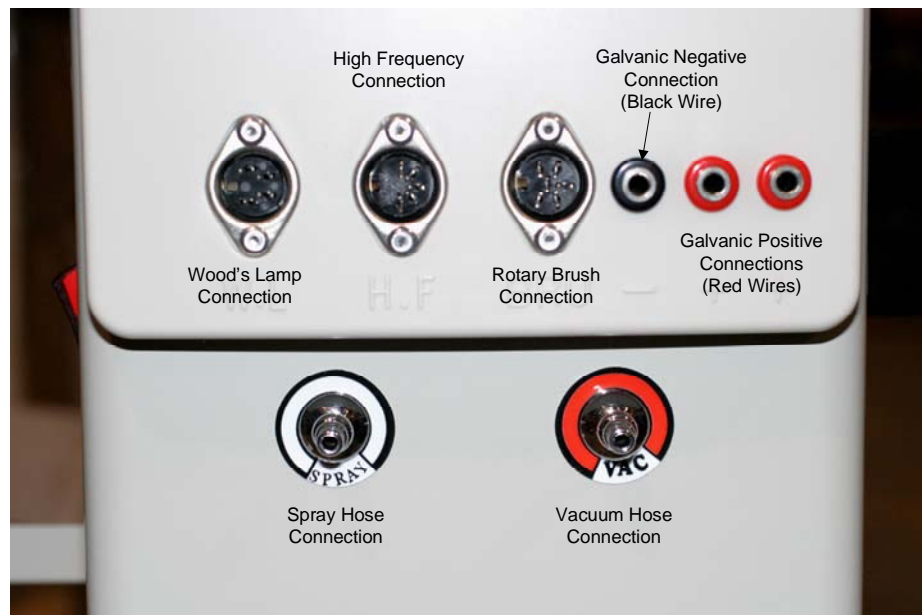

The smaller of the two shipping boxes contains the following items:

- ◆ Steamer unit
- ◆ Glass Steamer Jar and Gasket
- ◆ Measuring Cup



12. Remove the Steamer unit from the box. Slip the post of the steamer into the base unit.
13. Remove plastic wrapping and other packaging materials.
14. Inspect the unit for shipping damages.
15. The arm of the steamer is spring loaded. There is no adjustment knob.
16. Inspect the nozzle to make sure that no pieces of styrofoam have been caught in the nozzle.




17. Remove the cap to the water reservoir and inspect the opening to make sure nothing has gotten into the reservoir during shipping.
18. Inspect the Steamer Jar for nicks and/or cracks.
19. Prior to use, clean the Steamer Jar. Rinse it thoroughly and wipe dry.
20. Inspect the Gasket to make sure that it is not damaged.



21. Place the Gasket on top the jar. The groove in the Gasket goes over the top edge of the jar.



22. Loosen the Retaining Plate Knob located at the bottom of the steamer unit.
23. Move the Retaining Plate out of the way.

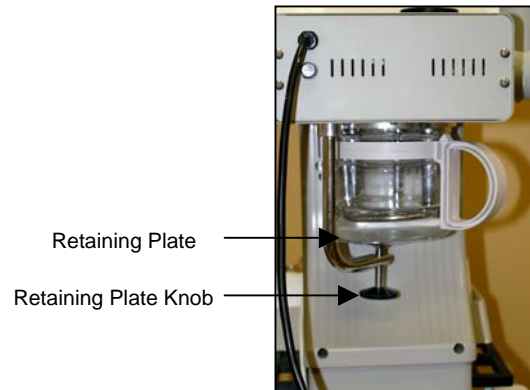

24. Carefully slip the Steamer Jar up over the heating element.



25. Move the Retaining Plate back under the Steamer Jar.
26. Tighten the Retaining Plate Knob. Do not over tighten.



27. Plug the Power Cord for the Steamer into one of the two outlets located on the right hand side of the stand just below the support bracket. (*Note: The Magnifying Lamp will be plugged into the other outlet.*)
28. Make sure the Timer on the front of the steamer is turned to zero.
29. Fill the measuring cup with distilled water **only**, and pour it through the opening at the top of the Steamer Unit. Repeat the process.



30. Assembly is complete. Plug in the Power Cord to the 8-in-1 DermiSystem and a working GFCI outlet.



31. You are now ready to begin offering facials to your clients!

