

CS² Models: Wellspring, Advanta, Alliance, & Portable Mat



Fig. 1



Fig. 2



Fig. 3



Fig. 4

To Set-up the Table:

- Place the folded table sitting on the rubber bumpers; with the heel of your hand, bump the tab between the face rest outlets and the table will open. (Fig. 1)
- Open the table part way, making sure the cables are still slack. (Fig. 2)
- Press on the long folding brace and pull on the legs at one end. (Fig. 3). Do the same on the other end. Open the table until the leg braces and table top are completely extended and unwrap any cables that may be caught on the legs.

To Set the Table Upright:

- When the table is fully opened, tilt the table upright until all four feet are on the floor. Lift one end of the table, press down on the middle and tension the cables a little. (Fig. 4)

To Open/Adjust/Close Tilt Backrest Section on Alliance:

- To Open/Adjust, release velcro pull tab located under the center at the end of the table and pull tab. Pull up on the center of the backrest, stopping at desired height (Fig. 5). Make sure that both sides of the tilting hardware are on the same setting and are both locked in place.



Fig. 5



Fig. 6



Fig. 7



Fig. 8



Fig. 9

- To close, raise the backrest fully and lower to closed position. Re-attach velcro tab to hold backrest in closed position. Refer to warning label on underside of table for precautions.

To Change the Table Height:

- Press button on the leg and rotate the leg extension slightly to take the button out of alignment with the holes, but still within sight. Adjust all extensions to the desired height and turn until the button pops out of the aligned hole. (Fig. 6)

To Close the Table:

- Turn the table on its side with the rubber bumpers down.
- For all models except the Advanta, store accessories as shown in Fig. 7. Store accessories in the Advanta as shown in Fig. 8.

Note: Advanta leg extensions must be fully retracted to close the table.

- Push the two halves of your table together to lock. (Fig. 9).

CAUTION

Dynamic Load Rating:

550 lbs.

Handcrafted in the U.S.A.

Inspected By: _____

Oakworks®
Original



All portable tables: U.S. Patent 5,009,170
CS System: U.S. Patent 6,192,809
Unilock Patent Pending

OAKWORKS®, Inc.

923 East Wellspring Rd., New Freedom, PA 17349

Toll Free Phone: 800-558-8850 Phone: 717-235-6807
Toll Free FAX: 877-562-4787 FAX: 717-235-6798

www.oakworks.com

