



# THERA°PEARL®

A NEW DEGREE OF RELIEF™

## What is THERA°PEARL

*It's Ice, It's Heat*

THERA°PEARL is an innovative hot/cold pack that is a drug-free way to relieve pain and inflammation associated with swelling from surgery, spa treatments and allergies, muscle strains and sprains, and a host of other pain-producing conditions.

## About THERA°PEARL Technology

*The Science of Soothing*

The Pearl Technology contained in every THERA°PEARL pack is game changing innovation born of an age old therapy. Crafted by a doctor who knew she could offer her patients something better than a bag of frozen peas, THERA°PEARL was designed with the consumer in mind. Whether chilled or heated, Pearl Technology™ conveniently holds its temperature for the doctor recommended time of 20 minutes while conforming to the body to deliver the perfect relief from aches and pains.

## Features

- ° Hot *and* Cold Therapy
- ° Freeze or Microwave
- ° Conforms to Body
- ° Reusable
- ° Hypoallergenic
- ° Non-Toxic
- ° No Mess
- ° Drug Free
- ° Drip Free
- ° No Odor

## Frequently Asked Questions

*What you should know*

**Q:** Can I heat the Eye-ssential?

**A:** Eyes are very sensitive, and you only get one set for life, so THERA°PEARL recommends that you be very careful when using our packs on your eyes. Be sure to test your heated pack before placing it on your eyes to make sure it is a comfortable temperature, and keep your eyes closed when using it. Some of the first Eye-ssential packs have a piece of metal attached to the strap that should be removed before microwaving.

**Q:** How do I clean my THERA°PEARL pack?

**A:** Clean your pack gently with mild soap, alcohol, or detergent and let stand for one minute.

**Q:** Can I take my THERA°PEARL straight from the freezer and heat in the microwave?

**A:** Yep. But be very careful not to overheat your pack. Test the pack temperature prior to application, as overheating may cause injury. Overheating may also damage or destroy your pack.





# THERA°PEARL®

A NEW DEGREE OF RELIEF™

## Product Comparisons

Features	THERA°PEARL	CVS Peas	Herbal Therapy Packs	Wheat/Rice Packs	ThermaCare	CryoMax
Hot & Cold			●	●		
Conforms to Body			●	●	●	
Drip Free		●	●	●	●	
Reusable		●	●	●		●
Consumer Friendly Packaging		●				
Bead Technology		●				
Easy to Disinfect		●				●
Non-Toxic		●	●	●		
Holds Temperature for 20 Mins		●			●	●

## Recommended Uses



### Eye-ssential Mask

°Use cold on eye brows for post waxing treatments

°Use warm on client while facing up during massage treatments

°Use during the mask portion of facial treatments



### Neck Wrap/Face Rest

°Use warm on face rest for when client is laying face down during massage

°Use cold on face to soothe skin after a chemical peel or microderm treatments

°Use warm as a relation treatment during waxing and mani/pedi treatments



### Sport Pack

°Use on shampoo bowl to add comfort for the client during shampoo

°Use on hands during mani/pedi services



### Round/Oval Pack

°Use cold for post waxing treatments on lips, bikini areas, and injections

°Use cold for post surgical relief for procedures such as Restylane, Botox, etc.



### Face Mask

°Replace steam towel treatments with the warm face mask

°Use cold for post surgical treatment relief (facial plastic surgery)



### Back Wrap

°Use during massage treatments

°Use warm for added relation on back and legs

°Use to compliment additional retail items during treatments



### Contour

°Use warm to comfort elbows and knees during treatments

°Use cold to soothe sore ankles, wrist, knees during treatments