



DIAMOND PEARLADHATION

CAUTIONS

- 1.To receive best results, you should clean face thoroughly before treatment.
 - 2.Do not apply water or other liquids to clean the device. Use dry cloth to wipe off residues instead.
 - 3.Do not use the cold treatment hammer on the following areas:
 - A. neuralgia areas
 - B. wounded areas
 - C. pregnant abdomens
 - D. body implants with metal pieces and silicon gel
- Keep skin dampened during the treatment. Dry skin may result in damages or ineffective treatment.

Contents

◆DIAMOND DERMABRASION.....	(1)
◆SUPERSONIC BEAUTY	(1)
◆ICE-PACK TREATMENT	(1)
◆BRIEF INTRODUCTION OF THE CONTROL PANEL.....	(2)
◆PARTS INTRODUCTION.....	(2)
◆DIAMOND DERMABRASION DEVICE WORKING PRINCIPLE..	(3)
◆QUALITIES OF USAGE.....	(3)
◆OPERATION PROCEDURE.....	(3)
◆SUPERSONIC DEVICE WORKING PRINCIPLE.....	(4)
◆SUPERSONIC BEAUTY OPERATION.....	(4)
◆SUPERSONIC BEAUTY PHOTO TREATMENT.....	(5)
◆CAUTIONS.....	(6)
◆ICE-PACK TREATMENT.....	(6)
◆OPERATION PROCEDURE.....	(6)
◆CAUTIONS.....	(7)

SKIN CARE ←

DIAMOND DERMABRASION



DIAMOND DERMA



Microdermabrasion, or particle skin rearing, is the latest technology for skin exfoliation. It has been adopted by leading American and European physicians and beauty spas/salons. The Envoy microdermabrasion uses a controlled vacuum to move sterile micro crystals to abrade the strata Corneum, which is the layer of dead cells on the skin then vacua need away after multiple treatments, this skin lines, lighten pigmented spots, imperfections.

◆ Diamond Dermabrasion

Treatment time: 5-10 minutes

This device is equipped with Diamond Dermabrasion pen, which is equipped with a diamond grade on pen head to abrade skin and rubdown stratum corneum. User can make this a decision on the speed of the treatment and the strength depending on the quality of skin.

◆ Supersonic Beauty

Treatment time: 10-20 minutes

This device uses supersonic waves in its treatment. Its oscillation is 1,650,000 times per second. With the help from supersonic and nani functions, oscillation can reach deep up to 3-7 cm of skin. So, the temperature of deep skin tissue rises up and accelerates the microcirculation process, which helps to raise the active metabolism of skin cells (with ointment this treatment can become more effective).

◆ Ice-pack Treatment

Treatment time: 5 minutes

This function is able to cool and easy skin cells. It also helps scared skin rejuvenate a faster.



Upper arm

- * From inside to outside
- * Move circle on outside



Hankers

- * Arch to ankle and up
- * Move on hankers up and down



Buttocks

- * Move up to down
- from side to middle



Waist, thigh

- * Waist to the center of back
- * Move clockwise on stomach
- * Move circle on hankers up and down
- * Move up and down on thigh

CAUTIONS

1. Cleanse face before supersonic treatment.
2. Spread enough face lotion on skin and switch on supersonic probe.
3. 15 minutes for all functions treatment. Overtime treatment will bring no more effect.
4. Lower temperature for face treatment.
5. Use small probe for small treatment area for a period of treatment is 8-10 minutes.
6. Spread face lotion on sensitive skin and move probe with small power. Redness caused by big power will fade away naturally.
7. There are 10 times for a period of treatment. You only can start another treatment in 7 days.
8. Do not space supersonic probe over 30 minutes to prevent burning and symptoms of such.

ICE-PACK TREATMENT

A) Massage function

Low-speed massage relaxes muscle and activates skin cells; high-speed massage can be used for catabolism

B) Cleansing function

28,000 times per second vibration makes fat breaking into water quickly and cleanses skin instantly.

Operation procedure

1. Prepare the device and skin-care products for treatment. And explain the treatment procedure to user so user become mentally prepared.
2. Plug cold-warm hammer into 4-pin socket of main unit and turn the power on.
3. Press "POWER" button.
4. Press "FACIAL" button.
5. Press "TIMER UP" button to increase treatment time and "TIMER DOWN" button to decrease treatment time. Display shows the time you set.
6. Press "FACIAL" button to stop.


DIAMOND DERMA

Supersonic beauty photo treatment



*To treat double chin, flabby neck and wrinkles work the utensil from middle to both sides (from chin up to ear).

Process around the abdomen:

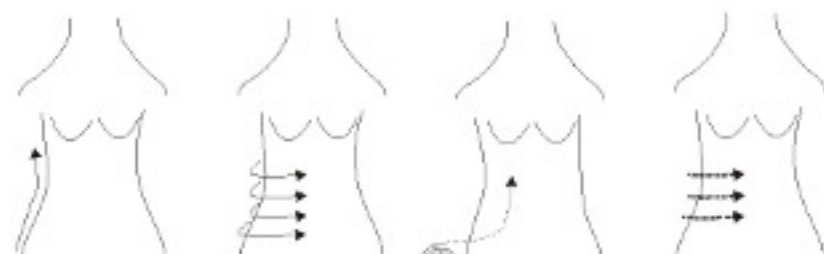

*Lay sideways move upward

*Lay sideways from side to middle

*Lay sideways move circle from hip to waist to stomach

*Lay sideways from coastal region to stomach



*Lay on back move circle from stomach up to chest


*Lay on back move circle on stomach then up chest

*Lay on back from rib to stomach

*Lay on back from stomach up to chest

(5)

※Brief introduction of the control panel

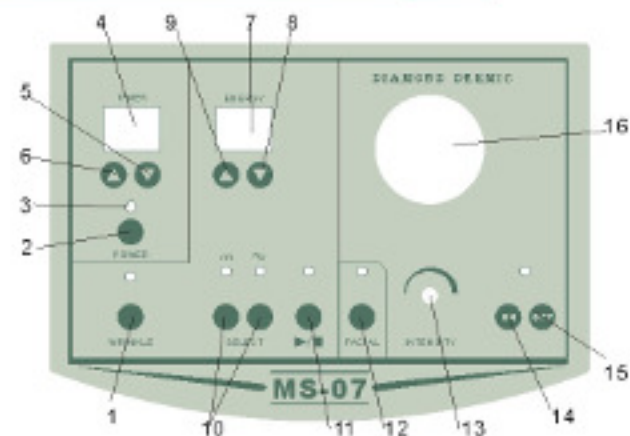

- 1.SUPERSONIC WAVE CONTROL
- 2.TURN OFF POWER
- 3.POWER INDICATOR
- 4.TIME DISPLAY
- 5.TIME DECREASE
- 6.TIME INCREASE
- 7.POWER DISPLAY
- 8.POWER DECREASE

- 9.POWER INCREASE
- 10.INTENSITY SELECT CONTROL
- 11.START/STOP
- 12.COOL TREATMENT CONTROL
- 13.INTENSITY CONTROL
- 14.START
- 15.STOP
- 16.BAROMETER

※Parts Introduction



(2)

DIAMOND DERMIC

Diamond Dermabrasion Device Working Principle


This device is equipped with diamond pen, which is equipped with a diamond granule on the pen head to abrade scar skin. It works with vacuum pump to control the power when doing skin treatment. So, this is a kind of physical treatment. It has many advantages, such as: stimulating skin with no sting, does not hurt skin, no side effects, no crystal granules, no coloring matter sedimentation, safe and clean.

Qualities of Usage

1. Suitable for all kinds of skin.
2. Advancing the quality of skin and rebuilding healthy skin.
3. Beautify skin that has rough scar skin, ageing scar skin or old stratum corneum.
4. Improves skin texture, decreases flabby skin and fixes coarse-pore skin.
5. Lighten speckles.
6. Get rid of pock, concave holes on skin.
7. No sting.

Operation Procedure

1. Insert the wire of diamond dermabrasion pipe into main unit interface.
2. Turn on the power (press "POWER" button).
3. Switch "DIAMOND DERMIC" button to "ON".
4. Adjust pressure value through turning the "INTENSITY" knob left or right. Its bar meter shows you the air pressure.
5. Set treatment time through pressing "TIMER UP" button to increase time and "TIMER DOWN" button to decrease.
6. Press "OFF" button to stop.

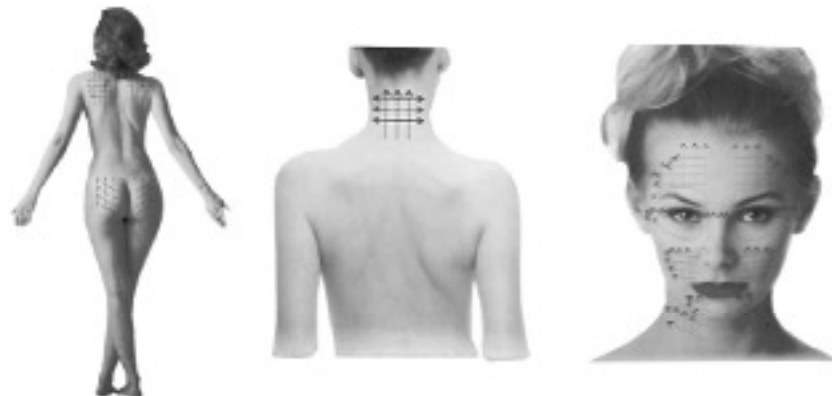


(3)

DIAMOND DERMIC

Supersonic Device Working Principle

1. Massage way

It is a way to get rid of fat from stomach, hip and thigh with the way of moving supersonic probe on skin at a speed of 0.5-2 cm per second. It lasts 2-3 second between two moving action. There should be 10cm between two lines draw by supersonic pen on skin.

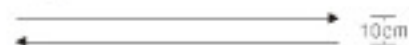

2. Circumrotate way

It is way to deal with thin place on face and arm at a speed of 0.5-2 cm per second. It lasts 2-3 second between two moving action. There should be 10 cm between two lines draw by supersonic pen on skin.



Supersonic beauty operation

1. Connect supersonic probe wire with the main unit.
2. Press "POWER" button.
3. Press "WRINKLE" button.
4. Press "ELECT" button to choose outlet wave, if "~~~~~" indicator lights it outlet interval wave, if "~~~~~" indicator lights it outlet continuous wave.
5. Press "ENERGY" button to strengthen power outlet.
6. Press "TIMER UP" button to increase time and "TIMER DOWN" button to decrease.
7. Press "WRINKLE" button once more, to turn off function.

0.5 W/cm² 1.25W/cm² and 5-15 minutes for face treatment more power and more time, not much more effect. For zone treatment, 3-5 times for a period. For chronic disease, such as discoloration, scar, pouches, black eyesocket and so on... 10 times for a period. Particular disease will get worse after 2-3 treatments. You should lessen intensity and expand the interval time. There are two kind of outlet wave: continuous wave and pulse wave. Pulse wave for signal, purulent skin tissue and expanded for capillary vessel skin treatment.

Face	Continuous wave	0.5-1.25W/cm ²	1.2-2.8W/cm ²
	Discontinuous wave	0.7-1.5W/cm ²	1.4-3.0W/cm ²
Eye	Continuous wave	0.5-0.7W/cm ²	0.75-1.5W/cm ²
	Discontinuous wave	0.6-0.9w/cm ²	1.0-1.8w/cm ²

(4)