



# PARAGON<sup>®</sup> F-I SKIN SCANNER

**ATTENTION** Recommended for use by professional, state-licensed estheticians. Operator shall be responsible for and damages to him/herself, the machine, and/or clients.

Before installing and/or using this unit, please carefully read these instructions. Be sure to keep this available for later reference.

## CAUTION

**BULBS CAN BECOME HOT DURING USE. CARE SHOULD BE TAKEN TO AVOID CONTACT BETWEEN THE SKIN AND BULBS.**

**ALWAYS TURN OFF THIS UNIT WHEN NOT IN USE. ALLOW THE UNIT TO COOL BEFORE CLOSING THE DRAPED DOOR.**

**NOTE: Cleanse the skin before commencing skin analysis.**

This device is intended for use during skin analysis. The use of filtered black light illuminates various skin conditions ordinarily invisible to the naked eye. The draped door allows you to perform analysis without the use of a darkroom. A concave mirror inside allows the client to observe the analysis.

### OPERATION

- A. Set the unit on a stable table or surface.
- B. Lift the draped door by pulling gently at the bottom. Be sure to lift the door up until it locks into position.
- C. Turn on the unit by pressing the power button.
- D. Instruct the client to position the head and neck inside the draped area.
- E. Using the viewing window to view the client's skin condition(s).  
NOTE: Use the color indicator chart as reference.

White Fluorescence	Thick corneum layer.
White Spots	Horny layer of dead skin cells.
Blue-White	Normal healthy skin.
Light Violet	Dehydrated/ dry skin.
Yellow/ Orange	Oily areas, comedones.
Pink	Oily areas, comedones.
Brown	Pigmentation problems.

- F. When analysis is complete, turn the unit off; wait for the bulbs to cool; close the draped door.

### TROUBLESHOOTING

When machine fails to work properly, use the following directions to detect the cause of the problem:

- A. Check the power cord for proper connection.
- B. Make sure the power switch is in the ON position.
- C. Check the power source by connecting another electrical appliance to the AC outlet.
- D. If the problem persists, please contact your dealer for assistance. To prevent an electrical shock, PLEASE DO NOT REMOVE ANY ELECTRICAL COVERS.

**WARRANTY**

- A. 1-Year warranty against manufacture defect.
- B. Wear-and-tear attachments, including but not limited to glass parts, not warranty item(s)

**Service**

This is a commercial use appliance, and all servicing should be performed by an authorized representative.

**Obligation**

In order to obtain warranty service, the product must be delivered to and picked up from the manufacturer at the user's expense, unless specifically stated otherwise in this warranty.

**Prolonged Storage**

If the appliance will not be used for a period of time, please empty all water/ lotion/ toners from containers and wrap the cord around the tower stand, store in a cool/dry area.

GARFIELD INTL. CO., INC.  
 15977 HERON AVENUE  
 LA MIRADA, CA 90638  
 PARAGON® is a trademark of Garfield International



Questions or Comments?  
 Please call 1-888-731-6688



[www.garfieldint.com](http://www.garfieldint.com)

**ATTENTION**

For your protection in the event of theft or loss of this product, please fill in the information below for you own personal records.

**Model No** \_\_\_\_\_ **Serial No.** \_\_\_\_\_  
(Located on back or bottom side of unit.)

**Date of Purchase** \_\_\_\_\_ **Purchase Price** \_\_\_\_\_

**Purchased From** \_\_\_\_\_