SILHOUET-TONE®

SILHOUET-TONE (CANADA) LTD

2185, Michelin, Laval (Québec)
H7L 5B8 Canada

Tél.: (450) 687-9456, 1 866 340-9456
Télé./Fax: (450) 687-5155
infost01@silhouettone.com

SILHOUET-TONE (U.S.A.) INC

1623 NW 84th Avenue
Miami, FL 33126

Tél. : 305.639.2100, 1.800.552.0418
stusa@silhouettone.com

SILHOUET-TONE (EUROPE) S.A.

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Tél.: 41.22.775.15.60
Télé./Fax: 41.22.775.15.67
info@silhouet-tone.com

*Sites: Tolhurst, Monterey, Michelin.

SILHOUET-TONE®

Code 48611248 2014-04-01

www.silhouet-tone.com

Imprimé au Canada / Printed in Canada

Thank you for purchasing this SILHOUET-TONE product.

This item is guaranteed for a period of two years. The enclosed guarantee card must be completed and returned to SILHOUET-TONE within 30 days from the purchase date.

This booklet is designed to familiarize you with this product. It explains its assembly, operation use and maintenance (as well as some precautionary measures). PLEASE READ INSTRUCTIONS CAREFULLY- This will save you time and money in unnecessary repairs.

Only qualified and licensed technicians should operate this unit. SILHOUET-TONE will not be held responsible for injuries or accidents resulting from improper use.

If you are unsure as how to operate the unit, do not hesitate to contact SILHOUET-TONE or your local distributor.

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This unit can be used for any small-scale depilation, with either warm or hot wax. It can be used for waxing the face, legs, arms, and back.

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Cleaning

- Heat wax until flowing freely. Empty container. Use a spatula or putty knife to empty any remaining wax.
- Never use any acetone-based products to clean unit.

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Precautionary measures

- The temperature at which wax melts and liquefies will vary from one type of wax to another. It is therefore impossible to determine in advance which thermostat setting will give maximum results. An adjustment period of a few days may be required before optimum results can be obtained from the Minipil.
- Before applying wax to any part of the client's body the technician must always test wax on her own skin to make sure that it is not too hot. The best place to test wax is the inside of the wrist.
- Wax must be at the correct temperature for use before client is seated for treatment. The Minipil should be plugged in ahead of time in order for the wax to be melted and ready for use. The required heating time depends on the quantity and quality of wax used.
- Stir the wax in the Minipil as often as possible, especially if wax has been recycled.
- If appointments are spread out, keep wax warm by turning down the intensity control (3).

IMPORTANT: ALWAYS remember to turn off unit when treatments are finished.

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Operation Instruction

- 1) Make sure that unit is switched to the "OFF" position (1) and intensity control turned to "0" (3).
- 2) When using lukewarm wax, place the wax can directly into the tank. If depilation is to be done with hot wax, put new hot wax *broken up into pieces) into wax container.
- 3) Switch on unit (1). Indicator light (2) should come on indicating that unit is operating.
- 4) Set intensity control (3) almost at maximum. Indicator light (4) will come on.
- 5) Let wax melt until it reaches a moderate temperature and adjust intensity control to the corresponding setting.
- 6) Unit is ready for depilation. Take wax from container or pan with a spatula.
- 7) After use, turn intensity control down to "0" and shut off unit (1).

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In case of problems:

The unit does not turn on	• Check if the unit is properly plugged in.
The wax container will not heat up	• Turn the thermostat control (3) down to minimum. The indicator light should go out. Then turn control to maximum. The indicator light should go on. A) If this does not occur, the thermostat must be changed. Contact your distributor or salesperson. B) If the indicator light goes on or off, there is a problem with the resistor (heating element). Contact your distributor or salesperson.

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After having checked all the above possibilities, if your unit is still not functioning properly, or not at all, contact your distributor or salesperson ONLY.

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Technical specifications CSA Approved

Voltage:	110 volts
Power consumption:	90 watts
Cycles:	50-60
Dimensions:	Diameter of small tank : 3.63" (9.20 cm) Diameter of large tank : 4.13" (10.47 cm) Depth of tank : 2.5" (6.35 cm) Width of device : 7" (17.78 cm) Height of device : 4.63" (11.75 cm) Depth of device (small) : 7.13" (18 cm) Depth of device (large) : 7.25" (18.41 cm)
Weight of device:	2.9 lb (1.32 kg)
Materials:	Steel, anodized aluminium, enamel coating

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