



RE-USE SOLUTION LOG CHART

i DID YOU KNOW that instruments that come into contact with mucous membranes or non-intact skin increase the risk of transmission of infectious pathogens and require a higher level of care when reprocessing?

- Step 1:** Pour fresh solution of Prevention™ HLD8 into clean soaking tray, no activation or dilution is required.
- Step 2:** Use AHPHLD8 test strips to validate the required concentration of hydrogen peroxide during the re-use period up to 21 days.
- Step 3:** Discard the solution up to 21 days, if the solution is visibly soiled, or if indicator strip fails test.
- Step 4:** Wash and dry instrument soaking tray before filling with new solution.

Product: _____

Lot Number: _____

Bottle Expiry Date: _____

Date Solution Poured

Re-Use Day	Date	Test Time	Test Results	Initial	Comments
1			Pass – Fail		
2			Pass – Fail		
3			Pass – Fail		
4			Pass – Fail		
5			Pass – Fail		
6			Pass – Fail		
7			Pass – Fail		
8			Pass – Fail		



Date Solution Poured

Re-Use Day	Date	Test Time	Test Results	Initial	Comments
9			Pass – Fail		
10			Pass – Fail		
11			Pass – Fail		
12			Pass – Fail		
13			Pass – Fail		
14			Pass – Fail		
15			Pass – Fail		
16			Pass – Fail		
17			Pass – Fail		
18			Pass – Fail		
19			Pass – Fail		
20			Pass – Fail		
21			Pass – Fail		
★		Make NEW Solution!			

Questions / Comments?
1-800-387-7578 | www.virox.com

