

Cavicide Implement Tray Kit

Before using the Cavicide System read all the instructions listed on the bottle thoroughly. It is a violation of U.S. Federal Law to use this product in a manner inconsistent with its labeling.

Usage

1. Used for a majority non porous surfaces and implements including plastic.
2. To pre-clean equipment or surfaces, spray directly. Remain on the surface for 30 seconds to pre-clean and 3 minutes for disinfection. Remove with a moist towel.
3. Remove the tray from the Cavicide system and fill to allow for complete submersion of pre-cleaned instruments and tray.
4. Turn tray handle in and close lid.
5. Allow implements to soak for a minimum of 10 minutes.
6. Remove, rinse, and dry.
7. Maintain implements in the sterile storage tray and cover or a sterile environment.
8. Change when the solution appears to be diluted or visibly soiled.
9. Maintain the date of solution using the sliding organizer on the outside of the tray system.