



HOT CABINET



STERILIZER



(small)

CATALOG

- STERILIZER..... (1)
- HOT CABINET..... (2)
- BREAKDOWN EXAMINATION & REMOVING..... (3)



OPERATING GUIDANCE

1. Turn switch ON, then put the instrument inside 5 minutes later.

2. The instrument is ready to use after 20 minutes.

3. Breakdown examination & removing.

4. You have to wear the protective glasses.

FEATURES

1. Having a stainless steel inner tray.

2. The door is opened by hand or automatically when you avoid any damage to the instrument.

3. Owning a piece of recording paper, which can prevent the ultraviolet ray from entering outside.

STERILIZER

COMPOSITION:


It is combined with one main unit, one two-level net, one handle, some screws etc.

FUNCTION:


A useful basic appliance with UV-tube for an optical sterilization of any kind of instrument, Ventose, surgical operation, brushes, surgical operation knives, clippers, rollers etc.

OPERATING GUIDANCE:

- 1) Turn switch " ON " , then put the instrument inside 5 minutes later.
- 2) The instrument is ready for use after one hour sterilization.

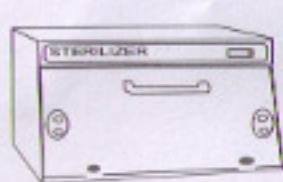

(small)

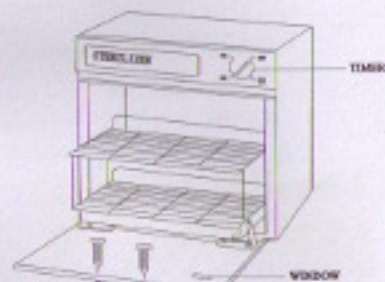

FEATURES:

- 1) Having an automatic switch, When the door is opened, the light will shut off automatically, which can avoid any damage to the skin and eyes.
- 2) Owning a piece of protecting alloy board, which can prevent the ultraviolet ray from radiating outwards.


HOT CABINET

FUNCTIONS:

- 1) maintaining temperature.
- 2) sterilizing
with a capacity of two dozen towels, The interior is furnished a constant temperature control of around 80°C.

OPERATING GUIDANCE:

- 1) Put towels inside.
- 2) Turn switch on five minutes later, the interior tank will reach up 80°C, Then the towels are ready for use.
- 3) Turn switch " OFF " when the work is over every day.
- 4) Pull the water-plate out from bottom of the cabinet, and pour the water off every day.

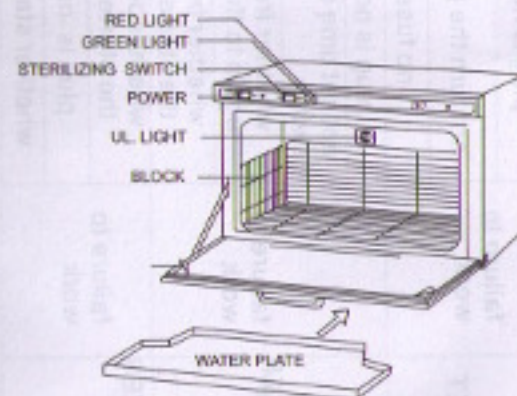

Breakdown Examination & Removing

Item	General malfunction	Examination	method of handling
HOT CABINET	failure to work	plug is not connected	insert plug into outlet
		not turn the power switch on	turn on the switch (press) then the indicator lamp will be lightening
		no fuse available	replace new one
STERILIZER	failure to work	plug is not connected	insert plug into outlet
		not set time or turn on power switch	to set time or turn on power
		whether front door is closed again to the case (main unit)	to close the front door and case then micro-switch can be started
STERILIZER (small)	failure to work	whether be available of the starter and lamp	replace the starter or lamp
		whether be available of the starter and lamp	replace the starter or lamp
		plug is not connected	insert plug into outlet
		whether start red switch	turn switch on

Attention: If the machine is not repaired by our company or retailers' technicians and is disassembled or assembled without permission, we aren't responsible for the consequence resulted.