

Instant Cold Pack

Usage

1. Shake contents to bottom of bag.
2. Locate inner pouch & squeeze.
3. Shake well to mix contents.
4. Apply to treatment area wrapped in a thin soft cloth.

**DO NOT APPLY LONGER THAN 20 MINUTES, MAY CAUSE FROSTBITE.
AVOID DIRECT CONTACT WITH SKIN.**

Warning

- ❑ If accidentally swallowed, give several glasses of milk or water and consult a physician.
- ❑ If solution contacts eyes, skin, or open wound, flood with water and call physician.
- ❑ Keep out of reach of children.
- ❑ Store at room temperature.
- ❑ For external use only.
- ❑ Do not put in freezer.
- ❑ Do not expose to heat.
- ❑ Discard if punctured or leaking.
- ❑ Dispose of properly after use.