



## INSTRUCTION MANUAL

## About Paraffin Therapy

The practice of heat therapy is centuries old. Through the ages, the Romans, the Japanese and many other cultures have realized the benefits of heat therapy on their minds and body. Hot paraffin treatments have been used for decades by doctors and physical therapists for those suffering from arthritis, bursitis and even for tired, swollen feet. Paraffin therapy reduces swelling in muscles and decreases inflammation of joints and connective tissues. Doctors have found that the therapeutic effects of paraffin treatments can last weeks longer than other heat treatments.

clean+easy® now brings this soothing, luxurious treatment into the salon as a conditioning cosmetic service for your clients. The Digital Paraffin Spa provides a soothing bath of warm paraffin that creates a heat-maintaining barrier for the skin. The paraffin coating holds the heat for 10-15 minutes, expanding the pores of the skin and allowing moisture to penetrate more readily.

For facials, manicures, pedicures and body treatment. Paraffin therapy provides every client with that extra special touch.




## Supply Checklist

Before you begin the therapy treatment, be sure you have all your supplies ready and available. Here's what you'll need:

- **clean+easy® Digital Paraffin Spa**
- **clean+easy® Paraffin Wax - 8 lbs.**
- **clean+easy® "Purify" Antiseptic Spray**
- **clean+easy® Paraffin Brush**
- **clean+easy® Paraffin Protectors**
- **clean+easy® Hand and Foot Insulating Mittens & Booties**
- **clean+easy® "restore" Dermal Therapy Lotion**
- **Towels**
- **Disposable blankets**
- **Electric blanket**

### Digital Control Panel

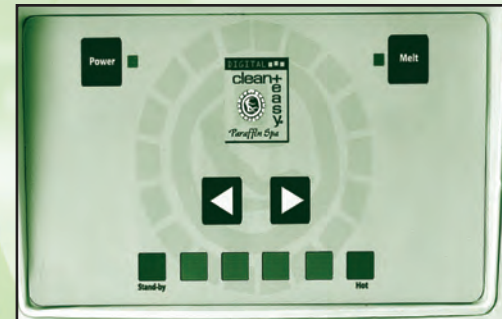

**Important:** Please read ALL instructions before you begin using the unit. Proper care and use of the unit will ensure a long life and trouble-free operation.

## Operating Your Digital Paraffin Spa

Your Digital Paraffin Spa is designed for use with a standard 120 volt outlet. The unit maintains paraffin at the ideal temperature for salon treatments. The Digital Paraffin Spa programmable "Auto-On/Off" timer may be left on continuously so you do not need to wait for paraffin to melt.

**The room temperature needs to be higher than 63°F for the unit to heat properly.**

### To Operate in Manual Mode

1. Remove the included 8 pounds of clean+easy® paraffin from the plastic bag and place into the Paraffin Spa, making sure the foot grade is on top of the paraffin, not below, then replace the lid.
2. Plug the power cord into a properly grounded, 3-prong electrical outlet and press the **"POWER"** button to turn the unit on. The unit is now in stand-by mode.
3. Press the **"Melt"** button to melt paraffin. The red indicator button, located next to the **"Melt"** button, will be on and all LED lights will race during the **"Melt"** mode.
4. Paraffin should completely melted within 2-3 hours. Do not use the spa at any time during the melt cycle. It is very hot and may cause discomfort to the skin.
5. When the wax has reached the optimum temperature, the green indicator lights are continuously on. This means the unit is ready to use. It is now safe to dip your clients.
6. To raise or lower the wax temperature, press the **"<"** or **">"** (**"UP"** or **"DOWN"**) buttons. The green LED light for the desired temperature will blink while the unit is changing temperature. Once the desired temperature is achieved, the corresponding green LED light will be continuously **ON**.

*Caution: If you press the > (UP) button until the Red LED blinks, the wax temp will not increase. It will instead return to 128F.*

## Stand-by Mode

Stand-by Mode is used to maintain the paraffin wax in a semi-hardened state, so that melting time is reduced to 30-40 minutes rather than the 2-3 hours required to melt hardened wax.

### Use Stand-By Mode Overnight

To set your clean+easy® Digital Paraffin Spa to **"STAND BY"** mode prior to leaving for the evening, press the **< "DOWN"** button until the blue **"STAND BY"** LED light is blinking. Once the wax reaches the **"STAND BY"** temperature, the blinking blue light will remain continuously **ON**. To re-melt your paraffin in the morning from **"STAND BY"** mode, press the **> "UP"** button until the red **"HOT"** LED light is lit. This process should take 30-40 minutes.

The clean+easy® Digital Paraffin Spa is ready to use when the center green LED lights are lit continually.

Never use the unit if any of the lights are blinking.

**Hint:** It is always suggested to have the paraffin spa **"OFF"** during the holidays or if unattended for more than one day.

## Adding Paraffin

1. Use only clean+easy® paraffin wax in your spa. clean+easy® paraffin wax is especially formulated to melt at low temperatures, **COMMERCIAL PARAFFINS CANNOT BE USED IN THIS UNIT.**
2. As you use your paraffin spa, the paraffin should be replenished. Use the convenient 1 lb. refills of clean+easy® paraffin wax to maintain the proper paraffin level. Paraffin should never be more than 2 inches from the top of the tub or above the **"MAXIMUM WAX LEVEL"** line posted inside the tub.
3. Leave your unit **"ON"**. Carefully add the 1 lb. refill to the reservoir. The refill will slowly melt. You may continue to do treatments as the refill melts.

## Paraffin Therapy Treatments

The clean+easy® Digital Paraffin Spa has several beneficial uses for any salon. A paraffin manicure will leave the hands feeling soft and gives every client a soothing, relaxing salon experience.

Paraffin therapy is also excellent for pedicure clients. The therapeutic heat treatment relaxes sore, tired feet, stimulates circulation and allows for a much more effective massage experience.

For facial clients, a paraffin treatment will leave your client's skin soft and rehydrated, while aiding in the infusion of any treatments into the skin.

As an added service for those who want an all over paraffin treatment, a full body paraffin wrap is recommended to deeply relax the muscles and soften the skin creating a mini-sauna on your body. Its therapeutic properties help release muscle tension, decrease joint stiffness, increase circulation and improve nutrition of the tissues.

## *Paraffin Profiles*

Place a clean+easy® Digital Paraffin Spa in your salon and watch your profits grow. Paraffin wax treatments are a great way to maximize skin care services and attract new clients.

Your clean+easy® Digital Paraffin Spa will quickly pay for itself as treatments cost you less than \$1 per client, but you earn from \$5 to \$15. As an added benefit, this easy treatment demands only a few minutes of your time, yet your clients will leave the salon feeling more pampered and relaxed than ever before. Client's referrals will soar!

## *Do's & Don'ts*

- DO** Plug your unit into a properly grounded, 3-prong electrical outlet.
- DO** Keep the paraffin level about 2 inches from the top of the tub.
- DO** Test the paraffin temperature by applying a small dab of paraffin wax on the backside of your wrist to make sure it is comfortable.
- DO** Protect your client's clothing and remove all jewelry. We suggest a gown or apron.
- DO** Make sure client's hands or feet are cleaned with "Purify" antiseptic spray and dry before beginning paraffin therapy.

- DO** Always guide your client's hands or foot into the paraffin spa. This maintains contact and prevents any mistakes on their part.
- DO** Dip foot into the paraffin spa up to the ankles and keep the foot in the same position as going in and coming out so you don't break the paraffin seal.
- DO** Dip hands all the way into the paraffin up to the wrist and keep hands in the same position going in and coming out so you don't break the paraffin seal.
- DO** Clean your paraffin spa regularly.

- DON'T** Use the paraffin spa if your client has impaired circulation or any skin irritation, rashes, cuts, burns or warts. This includes severe diabetes.
- DON'T** Heat paraffin on a stove or in a microwave.
- DON'T** Put used paraffin back into the spa. Keep the paraffin in the unit fresh and sanitary for your next client.
- DON'T** Heat other waxes, lotions, water, or liquids of any kind in your Digital Paraffin Spa. It is intended exclusively for use with paraffin wax.

## *Manicure Paraffin Treatment*

We recommend you apply a "restore" dermal therapy lotion after the manicure when hands and nails are at their cleanest. Your clients will leave the salon with a wonderful fresh feeling.

### **Instructions**

1. Complete manicure. **DO NOT APPLY POLISH!**
2. Spray hands with "Purify" antiseptic spray.
3. Apply a light coat of "restore" dermal therapy lotion.
4. Hold client's arm just above the wrist with the hand in a relaxed position, fingers slightly apart and palms down. Now gently immerse the client's hand into the paraffin wax up to the wrist.
5. Remove hand out of the paraffin wax and hold over spa unit for a few seconds until paraffin wax no longer appears shiny.
6. The hand should be immersed into the paraffin spa five times to form a thick paraffin glove.



## DO NOT LEAVE HAND IN THE PARAFFIN SPA TO SOAK

7. Slip hand into plastic protector bag. Press the back edge of the bag into the paraffin at wrist to secure it. Then slide on the terry mitt. This will retain heat for a longer treatment.
8. Repeat steps 4-7 with the other hand.
9. After 8-10 minutes, remove the terry mitts. For the cleanest and easiest paraffin removal, leave the plastic protector bags on.
10. Gently massage the hand through the plastic bag to loosen paraffin. Then, slide the paraffin off the hand. Dispose of the bag with paraffin after removal. Repeat with the other hand.
11. Massage hands until oil or lotion are completely absorbed.
12. Clean nails with polish remover to eliminate any paraffin wax or lotion residue prior to applying polish.
13. Nails will be warm, proceed with polish application.

## Pedicure Paraffin Treatment

Paraffin Therapy is the perfect ending to any pedicure service, leaving your clients feeling exceptionally pampered and relaxed.

### Instructions

1. Complete the pedicure. **DO NOT APPLY POLISH!**
2. Spray the feet with "Purify" antiseptic spray.
3. Apply a light coat of "restore" dermal therapy lotion to ankles and feet.
4. The paraffin spa should be placed with the deepest end towards the client. This allows the foot to be immersed in a natural position, with the heel lowered and the toes slightly upward.
5. Holding the client's foot above the ankle, immerse it into the paraffin spa unit up to ankle. Then remove.
6. Wait a few seconds until the paraffin wax no longer appears shiny and repeat. The foot should be immersed into the paraffin spa five times.
7. Slide foot into plastic protector bag. Then into a terry bootie. This will retain heat for a longer treatment.
8. Repeat steps 4-7 with the other foot. Leave the booties on for 10-12 minutes.

9. Remove terry booties. Leave the plastic protector bag on.
10. Gently massage the foot through the plastic bag to loosen paraffin. Then, slide the paraffin off the foot. Dispose of the bag with paraffin after removal. Repeat with the other foot.
11. Massage residual lotions or oil into feet and legs.
12. Clean toenails with polish remover prior to applying polish.

## FOR SANITARY PURPOSES, IT IS BEST TO USE A SEPARATE UNIT FOR PEDICURES.

## Body Paraffin Treatment

Body paraffin wrap is an effective therapeutic body treatment. It increases blood circulation, muscles and joints are left relaxed and the skin is intensely softened and moisturized. The client leaves with an invigorating and stimulating feeling all over.

### Instructions

1. Line the table with an electric blanket, then a disposable sheet.
2. Apply "Purify" antiseptic spray to your hands, mist the client's hands and feet, and wipe dry.
3. Apply "restore" dermal therapy lotion to the client's neck, shoulders, elbows, knees and feet.
4. Dip the paraffin brush into the paraffin spa, scooping enough paraffin on the brush to cover large areas. Use a folded paper towel under the paraffin brush as you are going from the spa unit to the client to avoid dripping paraffin onto the client.
5. Start at the ankles; brush on the paraffin quickly in long strokes, working upward to cover the body with an even coat of paraffin.
6. Apply at least five layers of paraffin. A thicker layer of paraffin will better moisturize the skin and allow for easier paraffin removal.
7. When finished coating the legs, cover them with a plastic wrap and towel to retain the heat while you continue.
8. When paraffin application is completed on the legs and back, have the client roll over to the other side and repeat steps 4 through 7.
9. Fold the disposable sheet around the body, and then fold the electric blanket over the sheet.
10. After 15-20 minutes, unwrap the body and remove the paraffin in large pieces.

11. Wipe off any paraffin residue on the body with a towel.
12. Discard all used paraffin.
13. Finally, massage "Hydrify" massage therapy cream to the shoulders, elbows, knees, and heels

## Facial Paraffin Therapy

Paraffin is applied after a basic facial is completed. The heat and oils present in the paraffin will open pores, allowing treatment creams, oils, or lotions to penetrate deeper into the skin.

### Instructions

#### DO NOT USE ALCOHOL-BASED PRODUCTS

1. Perform complete facial, analyzing skin to determine the best creams, oils, or lotions to apply prior to paraffin masque.
2. Use a headband and wrap the client's head with a towel to protect the hair.
3. Apply your choice of cream, lotion, or essential oils to face and throat.
4. Place cotton ball or pad over the eyes.
5. Check paraffin temperature on the backside of your wrist.
6. For strip use, dip a 2" wide gauze strip (about 6" in length) into the paraffin and place it on the face. The warm paraffin will hold the gauze in place.
7. For masque use, place the gauze mask over the face. Use a facial masque brush to apply a thin layer of paraffin over the face. Wait a few seconds until the paraffin no longer appears shiny, and then apply a second, third and fourth layer of paraffin. Keeping the layers of paraffin wax as even as possible.
8. Allow the client to relax for 10-15 minutes.
9. Remove the masque by simply lifting the edges of the gauze at the neck, and gently peel the mask off.
10. Dispose of the paraffin wax properly.
11. Gently massage the face. Remove any excess lotion or cream with a tissue.

## Cleaning your Digital Paraffin Spa

Your Digital Paraffin Spa should be cleaned after every 25 paraffin therapy treatments or when the paraffin is no longer clear and sediments have accumulated on the bottom of the spa. Old paraffin should never be reused.

### Instructions

1. Turn the paraffin spa **"OFF"**, then remove the lid and unplug. Allow paraffin to solidify.
2. After paraffin has completely hardened, plug in your spa unit. Turn unit **"ON"** to put in manual mode and adjust the temperature to 140 °F until the edges of the paraffin are softened and loose around the sides of the paraffin spa unit (about 15 minutes).

#### DO NOT PRY HARDENED WAX OUT OF PARAFFIN SPA OR USE METAL TOOLS TO SCRAPE BATH!

3. Push down on one end of the paraffin cake and it will tip up. Lift the paraffin out of the bath and dispose of it.
4. Turn the paraffin spa **"OFF"** and unplug it.
5. Wipe the inside of the paraffin spa unit with a disposable towel to remove residual paraffin wax.
6. Clean the inside and outside of the paraffin spa with an all-purpose cleaner. Thoroughly wiping the spa dry.

#### DO NOT IMMERSE PARAFFIN SPA IN WATER!

## Trouble shooting

Should you have any problems or questions regarding operation of your Digital Paraffin Spa, please write to us: American International Industries, 2220 Gaspar Ave., Los Angeles, CA 90040 or contact us via our website address [www.aiibeauty.com](http://www.aiibeauty.com). Telephone: 800-621-9585

## Important Safeguards

When using electrical appliances, especially when children are present, basic safety precautions should always be followed. **READ ALL INSTRUCTIONS BEFORE USING. KEEP AWAY FROM WATER. Electrical parts are electrically live even when the switch is off.**

1. Plug unit into a properly grounded 3-prong electrical outlet.
2. Do not use while bathing.
3. Do not place or store appliance where it can fall or be pulled into a tub or sink filled with water.
4. Do not place in or drop into water or other liquid.
5. If an appliance falls into water, unplug it immediately. **DO NOT REACH INTO THE WATER.**

**WARNING:** To reduce the risk of burns, electrocution, fire or injury to persons:

1. Do not leave appliance unattended while it is operating.
2. Close supervision is necessary when this appliance is used near children or invalids. **KEEP AWAY FROM CHILDREN!**
3. Use this appliance only for its intended use as described in the Digital Paraffin Spa Instruction Manual. The use of attachments not recommended by the manufacturer and may be hazardous.
4. Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged in any manner, or dropped into water.
5. Keep the cord away from hot or heated surfaces.
6. Never use while sleeping or while drowsy.
7. Do not drape the cord over a counter or a tabletop such that it can be pulled on by children or tripped over unintentionally.
8. Do not make contact with hot metal surfaces.
9. Use this appliance on a level surface.
10. Unplug from the outlet when not in use. Allow to cool before handling, cleaning, or storing the appliance.
11. Test wax on a small area of skin for warmth and possible skin reactions before using. If you are susceptible to allergic reactions or if you have circulatory problems, it is recommended that you consult a physician before using this appliance.

## NOTES

Item # 43505  
American International Industries  
Los Angeles, CA 90040  
clean+easy® is a trademark of  
American International Industries  
Made and printed in China





**clean+easy® DIGITAL PARAFFIN SPA OWNERSHIP REGISTRATION**  
**Warranty is not activated until card is received.**  
**Return card immediately.**

Name \_\_\_\_\_

Address \_\_\_\_\_

City \_\_\_\_\_

State \_\_\_\_\_ Zip \_\_\_\_\_

Phone \_\_\_\_\_ E-mail \_\_\_\_\_

Purchased from \_\_\_\_\_

Store Name \_\_\_\_\_

Date Purchased \_\_\_\_\_

Serial Number \_\_\_\_\_



**LIMITED SIX MONTH WARRANTY**

clean+easy® and American International Industries, Inc. warrants this unit to be free of defects in workmanship and materials for six months. This warranty does not apply to defects in appearance or damage resulting from accident, misuse, neglect, alterations to the unit, or repairs performed other than by clean+easy®. clean+easy® shall in no way be responsible for any incidental or consequential damages arising as a result of proper or improper use, prescription or maintenance of this product. In no event shall clean+easy be liable for such damage that exceeds the purchase price paid by the purchaser.

**TO OBTAIN SERVICE:**

For servicing outside USA, consult with your local dealer for details.

**IMPORTANT:** Do not tamper with or attempt to take this appliance apart. Such action will void this warranty.

**SAVE THIS FOR YOUR RECORDS**



**Postage  
Needed**

**clean + easy®**

Customer Service

2220 Gaspar Ave.

Los Angeles, CA 90040-1516

Attn: Warranty Dept