

Rotary Brush



I N S T R U C T I O N M A N U A L
C 9 2 1 1 T

© Universal Companies, Inc. 2007
All rights reserved for copyright purposes.



No reproductions by print or photocopies are allowed without written permission.

Rotary Brush Unit


The Rotary Brush Unit is used for procedures on the body as well as the face. The variety of brushes adds to the unit's versatility. The main purpose of the rotary brush is to deep-cleanse the skin to remove dirt, excess sebum and impurities from the surface and pores of the skin, as well as light callous removal on the hands and feet. With the proper cleanser and attachment, the brush unit can be used to achieve light exfoliation. The massaging action of the brush relaxes the client and creates a feeling of deep cleansing and pampering.

The rotary brush unit can be used with warm water or any type of cleanser, including cleansing milk, gels, and foaming cleansers.

Parts/Accessories

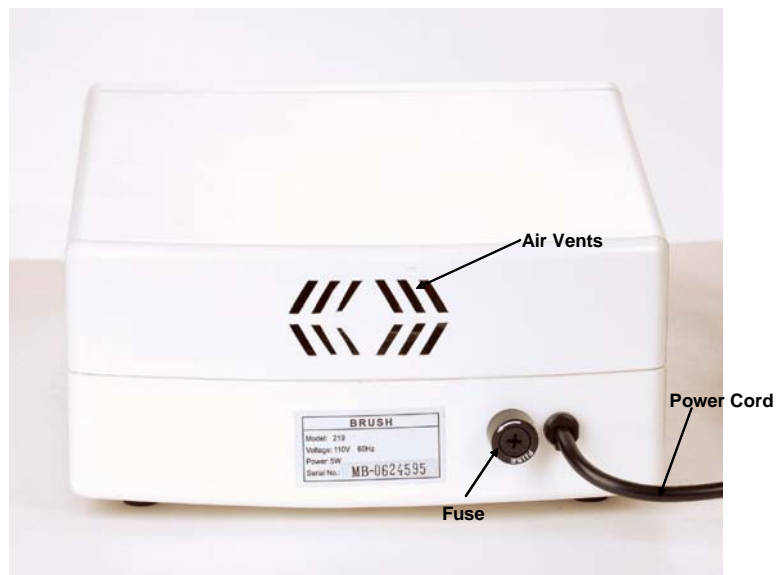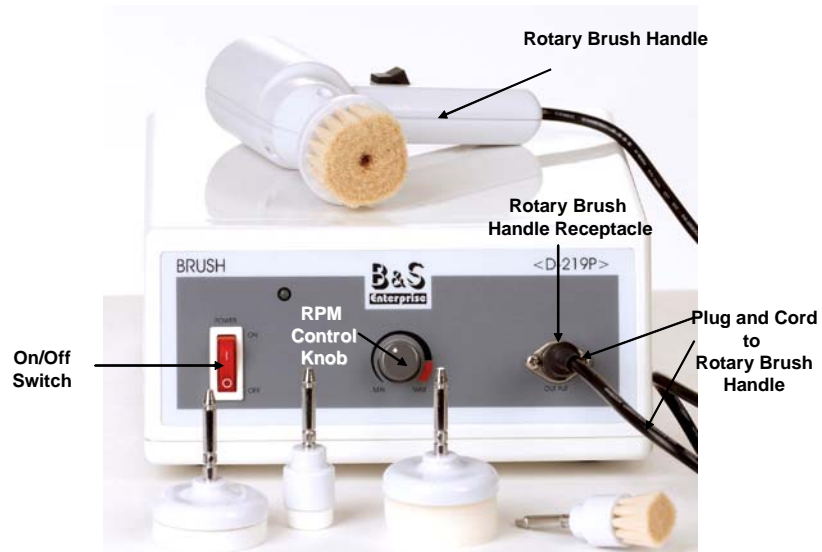

Rotary Brush Handle

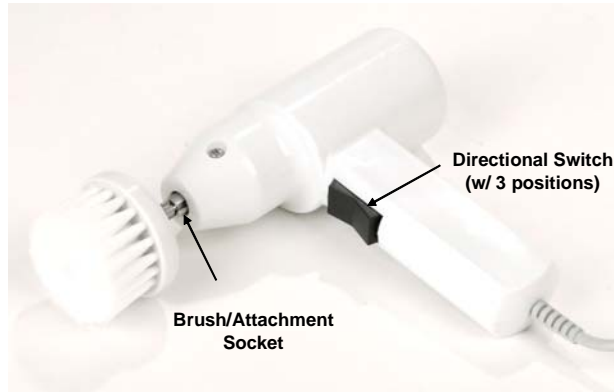

Attachments and Their Uses

- ❑ **Large Sponge Attachment** – It is frequently used at low speeds with toners. It can also be used on skin that is ultra-sensitive. (Note: The sponge is very porous and must be cleaned carefully to remove/destroy bacteria.
- ❑ **Foam Attachments** – The foam attachments are not to be used on the face. They are frequently used on dry areas, i.e. dry elbows, ashy knees.
- ❑ **Large Goat Hair Brush** – It is used for larger areas of the skin, i.e., the décolleté, chin, neck, cheeks, and forehead. It is also used for back treatments.
- ❑ **Small Goat Hair Brush** – It is used for smaller, more sensitive areas, i.e. around the nose, the mouth, eyes, etc.

Operations/Precautions

FOR PROFESSIONAL USE ONLY

To reduce the risk of electric shock:

- ❑ Always unplug the unit when not in use.
- ❑ Do not use while bathing.
- ❑ Do not place (or drop) into water or any other liquid.
- ❑ Do not reach for a unit that has fallen into water. Unplug immediately.

To reduce the risk of burns, electric shock, fire, or injury:

- ❑ Do not leave the Rotary Brush unit unattended when in operation.
- ❑ Do not allow children to operate the Rotary Brush unit.
- ❑ The Rotary Brush unit should be used only for its intended purpose as described in this manual.
- ❑ Do not use attachments that are not recommended by the Rotary Brush manufacturer.
- ❑ Do not operate the Rotary Brush unit if:
 - It has a damaged cord or plug.
 - If it is not working properly.
 - If it has been dropped or damaged.
 - If it has been dropped into water.
- ❑ If damaged, return the Rotary Brush unit to an authorized service center for examination and repair.
- ❑ Keep the cord of the Rotary Brush unit away from heated surfaces.
- ❑ Never block the air vents of the Rotary Brush unit. Do not place it on a soft surface such as a bed or couch where air vents may be blocked. Keep the air vents free of lint, hair, etc.
- ❑ Do not use the Rotary Brush unit outdoors.
- ❑ Connect the Rotary Brush unit to a properly grounded outlet.

Cleaning Precautions

- ❑ Do not clean the attachments in an UV sterilizer as this will discolor the plastic and shorten the life of the natural bristles.
- ❑ Use the unit with care to prevent damage to the motor inside the handle.
- ❑ Clean brushes after each use:
 - Sanitize with hot, soapy water.
 - Soak in a hospital-grade disinfectant for 10 minutes.
 - Rinse and blot dry.
- ❑ Clean the surfaces of the rotary brush handle using a hospital-grade disinfectant or 90% alcohol.
- ❑ Do not immerse the rotary brush handle in any liquid.
- ❑ Do not impede brush rotation while power is “On”.
- ❑ Do not pull or twist the cord. To unplug the unit, grasp the plug at the outlet and pull gently.

Contraindications for Use

- ❑ Use the brush very lightly on skin that has broken capillaries.
- ❑ Do not use the rotary brush on skin with severely broken capillaries.
- ❑ Do not use on broken, cut, or bruised skin.
- ❑ Keep the brush away from hair.

There may be other unknown contraindications for this Product. For this reason, you should always recommend that your Client discuss the use of this Product as part of their spa treatment program with their health care professional before commencing treatment. Universal Companies, Inc., its officers, directors, employees and vendors cannot be responsible for any claims, damages, injuries or expenses arising from your Client’s use of this Product prior to discussing it with their health care professional or from disregarding the Product’s instructions, and you and your Client agree to hold Universal harmless from any such claims. We and our suppliers care about the health and beauty of your Clients. While we make every reasonable effort to maintain strict quality control and training standards, every client is unique and spa treatments do affect people differently. Unless you advise us otherwise, we have to assume that your Client has discussed your spa treatment plan with their health care professional and secured their approval. Doing this will make your Client’s spa experience safe and will be supportive of their overall emotional and physical well being. After all, that is the goal of us all.

Prior to Treatment

1. Plug the power cord into a working electrical outlet (preferably a GFCI outlet).
2. Plug the cord for the Rotary Brush Handle into the Rotary Brush Handle Receptacle on the front of the main Rotary Brush Unit. Be sure to line up indicator tab with the top of the plug. Do not force the plug.
3. Choose the attachment that suits the type and condition of the client’s skin, the area to be treated, and the objective of the procedure. (See page 4 for a description of the attachments and their uses.)
4. Insert the brush/attachment into the tip of the Rotary Brush Handle by lining up the tab on the brush/attachment with the grooves in the opening in the tip of the hand held unit. Push the attachment into the rotary brush handle until hear a click.

Prior to the First Use of the Rotary Brush

- ❑ When you receive your Rotary Brush, cleanse the brushes prior to the first use to remove any loose debris.

Instructions for Use

- ❑ Before using the Rotary Brush unit, the skin should first be cleaned with a cleanser and sponges or cotton pads.
 - ❑ Steam the face with the facial steamer to soften the dead skin cells. This will make the removal process easier. (Option – Use warm moist towels to soften dead skin cells.)
 - ❑ Select a brush and insert the base into the brush handle until you hear a “clicking” or “snapping” sound, locking the brush into place.
5. Apply cleansing product generously over the entire treatment area. Add warm water to the cleanser as needed.
 6. Moisten the selected brush or attachment with water and remove excess moisture by touching it against a spa towel.
 7. Place the selected brush or attachment against the skin before you turn the unit on. If you don't, it will splatter when the unit is activated.
 8. Turn the switch for the Rotary Brush to the On position. Turn the switch on the handle of the Rotary Brush either up or down. Turn the Min/Max knob to the right until you have achieved the desired speed.
 9. To avoid product splatter:
 - Set the speed control knob to its lowest setting and turn on the power at the control panel.
 - Then set the direction of rotation on the rotary handle.
 - Increase speed slowly as needed.
 - Use lower speeds for dry or sensitive skin and higher speeds for coarse or thick skin.
 10. **Using a light pressure**, hold the brush vertical to the skin. *If you press down too much, you may irritate the skin and you will definitely deform the brushes over time.*
 11. To reverse the rotation of the brush:
 - Turn the RPM control knob to the lowest setting.
 - Move the Directional Switch on the Brush Handle to the “Off” position.
 - Brush should come to a complete stop.
 - Move the Directional Switch to the opposite position it was being used previously.
 - Resume treatment, increasing RPMs slowly.



Right



Wrong

Note: The direction of the rotation is optional. Some skin care specialists rotate the brush clockwise to start the exfoliation process, then make another pass across the same area with the brush rotating counter clockwise to get a good exfoliation.

12. To end the procedure or change attachments:

- Turn the RPM control knob to the lowest setting.
- Place the Directional Switch on the Brush Handle to the “Off” position.
- Brush should come to a complete stop.
- Remove the brush/attachment using sufficient force to pull the attachment **straight out** of the Rotary Brush Handle. Make sure you have a good grip on both the handle and the brush.
- End the procedure or choose another attachment and resume the procedure.

Caution: Avoid using brushes on broken, cut, or bruised skin. Using light pressure, hold the brush vertical to the skin. Keep the brush unit away from hair.

Caring for Your Rotary Brush Unit

Following each use, clean the brushes by washing them with an anti-bacterial cleanser and warm water, and then place the brushes in hospital grade disinfectant for 10 minutes. Use water to moisten the brushes and soften their bristles. To remove excess moisture, hold a towel over the brush and turn the brush unit on at low speed. Dry the brushes facing up. *Do not place the attachments brush side down to dry. That position will deform the brushes.*

The Foam Attachment and Pumice Attachment must be cleaned carefully. Soak them in Barbicide and use a nailbrush to cleanse the crevices.

Clean the surface of the Rotary Brush Handle using a hospital-grade disinfectant or 90% alcohol. Do not immerse the handle in liquid.

Note: Do not sterilize the attachments in an UV sterilizer, because it will discolor the plastic and shorten the life of the natural bristles.

Recommendation: Rinse brushes in a light conditioner to keep the brushes soft and to eliminate any natural smell that can occur.

Hint: In order to avoid any down time in using the Rotary Brush, purchase 2 extra sets of brushes. Three sets of brushes will optimize your work schedule. One set will be ready at all times. You will be cleaning one set and using the third set.

Trouble Shooting

If you experience problems with the operation of the rotary brush unit, please consult this troubleshooting guide before calling technical services at 1-800-558-5571.

If the unit does not work:

1. Check to make sure the power switch is set in the “On” position.
Check the main power plug to ensure that it is plugged in to a working electrical outlet (preferably a GFCI outlet).
Check the fuse located on the back of the unit (to the left of the Power Cord). If necessary, replace the fuse with a 2-amp fuse.
2. Make sure the RPM control knob is turned to a position that will create rotation.

3. Make sure the directional switch is set to one of its “On” positions.
4. Make sure the rotary brush handle is securely attached to the main unit at the rotary brush handle receptacle.
5. Check fuse. Replace if needed. You will need a Phillip’s screwdriver.



Parts List with SKU's

PART NAME	PART NUMBERS (SKU'S)
Rotary Brush Handle	CP9422T
Small Goat Hair Brush	CP9424T
Large Goat Hair Brush	CP9425T
Large Sponge Attachment	CP9423T
Large Foam Attachment	CP9426T
Small Foam Attachment	CP9427T
5 amp Fuse	CP9428T

Specifications

Dimensions

- ❑ 9” wide, 4.25” high, 10.5” deep

Electrical

- ❑ 110 V
- ❑ 60 Hz
- ❑ 5 watts
- ❑ 5 amps max

Warranty Statement

The manufacturer (the “warrantor”) warrants to the original purchaser that this equipment will be free from defects under normal and proper operation for a period of one (1) year from the date of purchase. This warranty covers all parts and labor for that one (1) year period. This warranty shall not apply to any equipment that has been subjected to abuse, misuse, and alteration of any type or to any defect or damage caused by repair, replacement, substitution or use with parts other than parts approved by the manufacturer.

The manufacturer shall not be liable for the loss of use of this equipment, loss of time or business, inconvenience or any other indirect, consequential, special or incidental damages due to defects covered by this warranty or any implied warranty.

To implement this warranty, contact the dealer from whom you purchased the equipment. If necessary, you will be asked to return all, or the appropriate part, of the equipment. If warranted, the repairs will be performed at no charge and the equipment returned to you. Unit owner must prepay all shipping costs when returning items for warranty service.

This limited warranty is the only written or express warranty given by the manufacturer. This warranty gives you specific legal rights but you may also have other rights, which may vary from state to state or province to province.

Authorized Technical Service Center:

Universal Companies, Inc.
18260 Oak Park Drive
Abingdon, VA 24210
Phone: 1-800-558-5571
Fax: 1-800-237-7199
E-Mail: info@universalcompanies.com

DISTRIBUTED BY UNIVERSAL COMPANIES



Universal Companies, Inc., 18260 Oak Park Drive, Abingdon, VA 24210

Phone: 800.558.5571 **Fax:** 800.237.7199

Web: www.universalcompanies.com

Email: info@universalcompanies.com

C9211TM
