



PARAGON[®] 204 VACUUM & SPRAY

ATTENTION Recommended for use by professional, state-licensed estheticians. Operator shall be responsible for and damages to him/herself, the machine, and/or clients.

Before installing and/or using this unit, please carefully read these instructions. Be sure to keep this available for later reference.

CAUTION

VACUUM TREATMENT TIME VARIES, DIFFERENT SKIN TYPES REQUIRE DIFFERENT TREATMENTS. PLEASE USE AT YOUR DISCRETION. WHEN THE SKIN IS THIN, DRY, OR AGING, A LIGHT SUCTION IS RECOMMENDED. FOR OILY SKIN, IT IS RECOMMENDED THAT THE VACUUM TREATMENT BE PERFORMED AFTER THE DISINCRUSTATION TREATMENT WITH THE GALVANIC CURRENT.

WARNING: BEFORE PLUGGING IN THE UNIT OR TURNING THE UNIT ON/OFF, BE SURE TO RESET THE VOLUME CONTROL KNOB TO ITS MINIMUM SETTING. STARTING THE UNIT AT ITS MAXIMUM SETTING MAY DAMAGE THE FUSE.

The vacuum system aids deep pore cleansing and is also referred to as the suction machine. The suction removes out deeply embedded dirt, sedum and other impurities. Another benefit is a deep penetrating massage, drawing blood to the surface of the skin. Note: the vacuum suction does not dislodge blackheads.

Included with this unit should be a pre-assembled complete handle, 6 glass ventricles, and 2 spray bottles.

VACUUM OPERATION

- A. Connect the air tubing with the "yellow" adapter to the "yellow" connector on the machine. Also connect the tubing with the "green" adapter to the "green" connector.
- B. Select the appropriate ventouse and insert to the yellow adapter (a metal connector w/ a built in filter).
- C. Turn the green power switch to the ON position.
- D. Turn the VACUUM ADJUSTMENT KNOB this knob clockwise MAX. (This knob can turn approximately 1½-2 revolutions).
- E. The ventouse control hole should be covered when the ventouse is touching the skin and should be uncovered when taken off the skin.
- F. After the vacuum treatment is complete, use the spray to lightly cover the face with a toner, and pat dry with cotton pads or a clean towel. This will help remove any debris left on the skin and close the pores.

SPRAY OPERATION

The spray bottle can be filled with distilled water or toner. The spray bottle is not recommended to be filled with lotion; lotion is too thick and will cause the blockage of the very fine spray hole in the cap.

- A. Connect the "green" adapter to the spray bottle.
- B. Hold the bottle in an upright position.
- C. Place your finger over the small hole at the top of the bottle.

MAINTENANCE

Attachments should be sanitized after each use.


- A. Wash glass ventouses and attachments thoroughly with soap and warm water.
- B. Soak the glass ventouses in alcohol for 5 minutes allowing them to sterilize.
- C. Dry ventouses and store in sterilizer cabinet or autoclave.

TROUBLESHOOTING

When machine fails to work properly, use the following directions to detect the cause of the problem:

- A. Check the power cord for proper connection.
- B. Make sure the power switch is in the ON position.
- C. Check the power source by connecting another electrical appliance to the AC outlet.
- D. If the vacuum suction force is not strong enough, turn the VACUUM ADJUSTMENT KNOB clockwise all the way (approximately 2½ turns). If this does not work, try cleaning the metal filter within the yellow adapter by detaching the yellow connector and removing the filter (Figure 7.5).

FIGURE 7.5

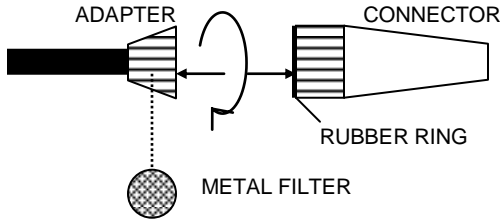

- E. If the spray bottle fails to spray, try using a small wire to clean the small passageway hole on the spray head.
- F. If the problem persists, please contact your dealer for assistance. To prevent an electrical shock, PLEASE DO NOT REMOVE THE COVER.

WARRANTY

- A. 1-Year warranty against manufacture defect.
- B. Wear-and-tear attachments not warranty item(s)

Service

This is a commercial use appliance, and all servicing should be performed by an authorized representative.

Obligation

In order to obtain warranty service, the product must be delivered to and picked up from the manufacturer at the user's expense, unless specifically stated otherwise in this warranty.

Prolonged Storage

If the appliance will not be used for a period of time, please empty all water/ lotion/ toners from containers and wrap the cord around the tower stand, store in a cool/dry area.

Variable Settings 120V, 144W, 1.20amps

GARFIELD INTL. CO., INC.
 15977 HERON AVENUE
 LA MIRADA, CA 90638
 PARAGON® is a trademark of Garfield International



Questions or Comments?
 Please call 1-888-731-6688



www.garfieldint.com

ATTENTION

For your protection in the event of theft or loss of this product, please fill in the information below for you own personal records.

Model No _____ **Serial No.** _____
(Located on back or bottom side of unit.)

Date of Purchase _____ **Purchase Price** _____

Purchased From _____