



PARAGON[®] 205 GALVANIC

ATTENTION Recommended for use by professional, state-licensed estheticians. Operator shall be responsible for and damages to him/herself, the machine, and/or clients.

Before installing and/or using this unit, please carefully read these instructions. Be sure to keep this available for later reference.

CAUTION

DO NOT USE THIS UNIT ON PEOPLE WITH PACEMAKERS OR CONTACT LENSES. PREGNANT WOMEN SHOULD CONSULT THEIR DOCTORS FIRST.

DO NOT USE THE GALVANIC CURRENT OVER AN AREA HAVING MANY BROKEN CAPILLARIES.

CLIENTS SHOULD REMOVE JEWELRY AND ALL METAL OBJECTS FROM BODY BEFORE TREATMENT BEGINS.

WARNING: BEFORE PLUGGING IN THE UNIT OR TURNING THE UNIT ON/OFF, BE SURE TO RESET THE VOLUME CONTROL KNOB TO ITS MINIMUM SETTING. STARTING THE UNIT AT ITS MAXIMUM SETTING MAY DAMAGE THE FUSE.

The galvanic current is a low level and direct current. Chemical changes are produced when this current is passed through certain containing acids and salts: (1) to soften grease and sebum deposits (2) to introduce water soluble treatment products into the skin.

Included with this unit should be a pre-assembled complete handle, 3 attachments, a rubber grounding pad with grounding pouch, and an adjustable Velcro wrist band.

INTRODUCTION

There are two poles or terminals, a negative (-) and a positive (+). Polarity determines the flow of galvanic current.

- A. Positive Pole (Anode)
 - a. Close the follicles (pores) after the facial treatment.
 - b. To decrease redness, as in mild acne.
 - c. To prevent inflammation after comedone and blemish treatment.
 - d. To force acid PH solution such as astringent into the skin.
- B. Negative Pole (Cathode)
 - a. Stimulate the circulation of blood.
 - b. Force disincrustation lotion into the skin.

OPERATION

- A. Wet the black galvanic pouch with tap water (distilled water is not as conducive) and slip the black rubber grounding disk into the pouch.
- B. Secure grounding disc and pouch to client's wrist with the Velcro wrist strap.
- C. Wet the area of skin to be treated with lotion or toner.
- D. Turn the switch to the ON position, and select the appropriate power according to your client's sensitivity. The current should be turned on to its lowest amperage before the attachment is placed on the client's face.
- E. Note: the client may experience a metallic taste in the mouth. A slight prickling sensation may indicate that the current is too strong. Adjust the power to the client's sensitivity.

MAINTENANCE

Attachments should be sanitized after each use.

- A. Wipe attachments with rubbing alcohol and a soft cloth.
- B. Dry attachments and store in sterilizer cabinet or autoclave.

TROUBLESHOOTING

When machine fails to work properly, use the following directions to detect the cause of the problem:

- A. Check the power cord for proper connection.
- B. Make sure the power switch is in the ON position.
- C. Check the power source by connecting another electrical appliance to the AC outlet.
- D. If the power lamp is not on, this may indicate that there is poor conductivity. Wet the grounding pouch with tap water and the skin to be treated with lotion or toner. (Refer to 9.3)
- E. If the problem persists, please contact your dealer for assistance. To prevent an electrical shock, PLEASE DO NOT REMOVE THE COVER.

WARRANTY

- A. 1-Year warranty against manufacture defect.
- B. Wear-and-tear attachments not warranty item(s)

Service

This is a commercial use appliance, and all servicing should be performed by an authorized representative.

Obligation

In order to obtain warranty service, the product must be delivered to and picked up from the manufacturer at the user's expense, unless specifically stated otherwise in this warranty.

Prolonged Storage

If the appliance will not be used for a period of time, please empty all water/ lotion/ toners from containers and wrap the cord around the tower stand, store in a cool/dry area.

Variable Settings 120V, 3.6W, 0.03amps
