

USER MANUAL- OAKWORKS SHIELD

MODEL: 82611 Oakworks Shield

USE: Oakworks Shield is designed to create a minimum distance of 6 feet between service providers and clients during treatment. It can also be used to separate clients from one another in multi-station rooms.



The shield can be placed anywhere between you and the patient, maximizing the distance aerosols must travel between you



Centering yourself and the patient on the shield side to side, with your heads relatively low is the best.



Minimize the gap between the table and the shield by adjusting the height of the shield or the table.



Reach under the shield or position the body part outside of the shield.



The shield can be used between patients to minimize exposure to aerosols and droplets. The shield is also easily moved when not needed.

OAKWORKS®

Phone: 717-235-6807 www.oakworks.com
923 East Wellspring Rd. New Freedom, PA 17349

USER MANUAL- OAKWORKS SHIELD

DIRECTIONS: Position the film of the Oakworks Shield so that it creates a minimal gap with the bed surface, and the extended film sheet is centered to the working area.

To change the height of the film, loosen the locking knob on the lower roll support and position it at the desired height, then tighten the knob. Be sure to support the weight of the roll when loosening the knob to prevent it from dropping.

Reposition the top cover so it is down against the top of the film roll.



To load a fresh sheet of film from the shield, first use the scissor to cut off the film sheet near the roll. Then remove the sheet from the clips and discard. Pull a new sheet from the roll and attach to the clips.



CLEANING AND DISINFECTION: Clean using a mild solution of 4:1 diluted non-alcohol cleaner such as Formula 409[®], Fantastik[®], or Green Windex[®]. To clean the clear film, a glass cleaner such as Windex[®] may be used.

Recommended Disinfectants (Approved US Products): Parker Labs Protex™ (EPA# 6836-152-82613), MADA Medical MadaCide-FD (EPA# 1130-15-11703), HealthLink[®] EZ-Kill[®] (EPA#59894-10-67454) or Clorox[®] 10% household bleach solution (EPA# 5813-100) following the Manufacturer's disinfection instructions. Note: Damage caused by unapproved substances will not be covered under the warranty.


OAKWORKS[®]

Phone: 717-235-6807 www.oakworks.com
923 East Wellspring Rd. New Freedom, PA 17349

USER MANUAL- OAKWORKS SHIELD

ASSEMBLY INSTRUCTIONS: The Oakworks Shield is shipped partially assembled. You will need a mallet or hammer to assemble it.

1. Remove all parts from packaging and ensure all parts are included.

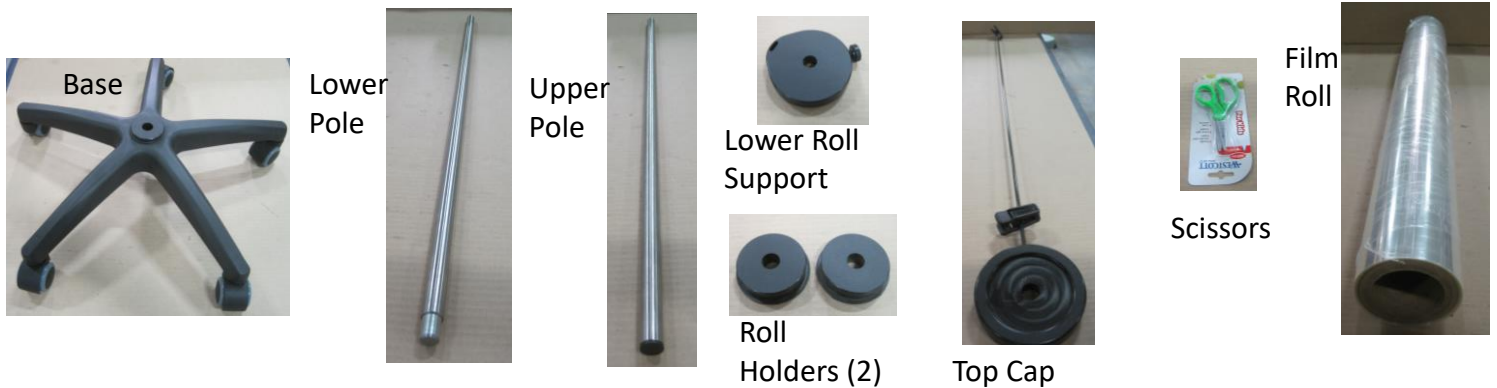


2. Place the unthreaded end of the lower pole into the hole in the center of the star base. Turn the assembly upside down and place the threaded end of the pole on a block of wood on the floor. Hammer down the bottom of the base to seat the pole into the base.



3. Turn the assembly right side up and thread the upper pole onto the lower pole.



4. Slide the lower roll support onto the pole and tighten the locking knob.

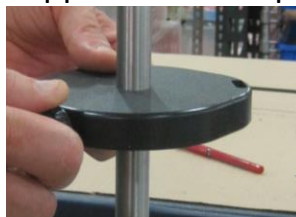

5. Slide one of the two roll holders onto the pole.



USER MANUAL- OAKWORKS SHIELD

ASSEMBLY INSTRUCTIONS (continued):

6. Slide the film roll onto the pole and seat it onto the roll holder.



7. Slide the second roll holder onto the pole and seat it in the film roll.



8. Slide the top cap onto the pole with the recessed side down, against the top of the film roll.



9. Place the scissors into the slot in the lower roll support.



REORDER FILM ROLLS: Order Part Number 82664

Original Instructions

Manual Part Number: SPA-RA-PM-153

Revision level: 1

Revision Date: 07/29/2020

English, Printed in USA

OAKWORKS®

Phone: 717-235-6807 www.oakworks.com
923 East Wellspring Rd. New Freedom, PA 17349