



UV STERILIZER MANUAL

ITEM: MSI-SAAPP-209A

Thank you for purchasing our Dermalogic UV Sterilizer. To use this product safely and correctly, please read this manual carefully and abide by all directions and notes.

Safety Instructions



1. Make sure the appliance is plugged into a sufficient outlet.
2. Children are not allowed to operate this appliance for safety concerns.
3. Place the appliance on a flat, level, and solid surface.
4. Never use any corrosive detergents to clean the appliance or its shelves.
5. DO NOT open or close the door with excessive force.
6. Keep the appliance away from explosive and flammable materials. Ensure that the distance between the appliance and the surrounding material is at least 30 cm.
7. Do not operate the appliance near water.
8. For your protection, please DO NOT stare at the UV light directly while the machine is operational.
9. Always ensure the door of the appliance is closed when it's operating.
10. Please shut off the unit after sterilizations have finished.
11. Use care when handling glass tools or accessories to avoid the risk of injury.

Function:

This product uses UVC light with Ozone to optically sanitize various instruments, including surgical tools, knives, scissors, brushes, etc.

Operation Instructions

1. Put the tools into the sanitizer and close the door. Press the POWER button to activate the unit.
2. Tools will be ready to use after 30 – 60 minutes of sanitizing.

Specifications

Voltage	110V/120V
Frequency	60Hz
UV Lamp Power Rating	8W
Capacity	8.5L

Troubleshooting Checklists

Issue(s)	Possible Cause(s)	Solution(s)
The appliance could not be turned on.	<ol style="list-style-type: none">1. Connection of UV light tube is loose.2. The fuse has blown.	<ol style="list-style-type: none">1. Twist the UV light tube in firmly.2. Replace the UV light tube.3. Replace the fuse.
The door could not be fully closed	<ol style="list-style-type: none">1. The screw on the door hinge is loose.	<ol style="list-style-type: none">1. Adjust the door position and retighten the screws