

## INSTRUCTIONS FOR USE:



**1.** Open the powder container and put on disposable gloves. Completely cover the spill with powder. Caution: Slippery when wet.



**2.** Use the scoop and detachable scraper to clean up the gelled spill along with broken glass, if any.



**3.** While still holding the scoop, remove glove by pulling cuff down over scoop leaving all waste, including the scoop and scraper, inside the inverted glove. Seal glove with tie and place into the disposal envelope.



**4.** While still wearing the second glove, remove disinfecting towelette from packet and unfold. With gloved hand, wipe surface, wetting thoroughly. Allow to air dry. Invert glove, leaving towelette inside. Seal with the sticker and dispose of properly according to CDC/OSHA infection prevention recommendations.