

UV Sterilizer Double Rack



I N S T R U C T I O N M A N U A L
C 9 2 2 3 T

© Universal Companies, Inc. 2005
All rights reserved for copyright purposes.

No reproductions by print or photocopies are allowed without written permission.



UV-Sterilizer

The UV Sterilizer uses an ultra-violet lamp to sanitize and maintain sterilization of implements using ultra-violet light including (but not limited to) any kind of instrument, surgical instruments, clippers, and rollers. The UV light can only sanitize what it can “see”, so hard surface implements are best suited for this type of sterilization. Porous surfaces such as towels, sponges or the inside of bottles may not sanitize properly, as the light may not shine on internal areas that can contain bacteria. The UV light comes on automatically when the door is closed and turns off automatically when the door is opened.

Sterilize instruments using a hospital-grade disinfectant before placing them in the UV Sterilizer.

Parts/Accessories



Operations/Precautions

FOR PROFESSIONAL USE ONLY

To reduce the risk of electric shock:

- Always unplug the UV Sterilizer when not in use.
- Do not place (or drop) into water or any other liquid.
- Do not reach for a unit that has fallen into water. Unplug immediately.

To reduce the risk of burns, electric shock, fire, or injury:

- ❑ Do not look at the ultraviolet light.
- ❑ Do not leave the UV Sterilizer unattended when in use.
- ❑ Do not allow children to operate the UV Sterilizer.
- ❑ Extreme caution should be used when UV Sterilizer is used near invalids.
- ❑ The UV Sterilizer should be used only for its intended purpose as described in this manual.
- ❑ Do not operate the UV Sterilizer unit if:
 - It has a damaged cord or plug.
 - If it is not working properly.
 - If it has been dropped or damaged.
 - If it has been dropped into water.
- ❑ If damaged, return the UV Sterilizer to an authorized service center for examination and repair.
- ❑ Keep the cord of the UV Sterilizer away from heated surfaces.
- ❑ Never block the air vents of the UV Sterilizer. Do not place it on a soft surface such as a bed or couch where air vents may be blocked. Keep the air vents free of lint, hair, etc.
- ❑ Do not use the UV Sterilizer outdoors.
- ❑ Connect the UV Sterilizer to a properly grounded outlet (preferably a GFCI outlet).

Instructions for Use

1. Sterilize the instrument(s) using a hospital-grade disinfectant.
2. Place the instrument(s) on one of the two wire racks located inside the unit.
3. Push the Power Switch to the “On” position.
4. Leave the instrument in the sanitizer for one hour.
5. Turn off the unit.
6. Leave the instrument(s) in the sanitizer until ready to use.
7. Open the door and remove the sanitized instrument.

Note: To maintain sterility, you should handle the instrument using a clean, sterile cloth or sterile gloves.

8. After sterilizing instruments, you may place them in one of Universal’s Sterilization Pouches (C413T) prior to putting them in the UV Sterilizer. Because instruments can be stored in the pouches, they (shown to the right) help maintain sterility.



9. Unplug the unit when not in use.

Trouble Shooting

If the UV Sterilizer is not working:

1. Make sure UV Sterilizer is set in the “Power On” position.
2. Check the main power plug to ensure that it is plugged in to a working electrical outlet.
3. If the UV Sterilizer is still not working, check the UV bulb and replace if necessary. (See instructions below.)

To change the UV bulb:

1. The UV Bulb is located in the upper rear portion of the cabinet of the UV Sterilizer.
2. To remove the bulb, turn it one-quarter turn in either direction until the pins line up with the opening on the lamp sockets. Remove it.
3. To replace the bulb, line the pins up with the opening on the lamp socket. Slip the bulb into place and then rotate it one-quarter turn until it locks into place.
4. If the unit still doesn't work, call Technical Services at Universal Companies at 1-800-558-5571.

Parts List with SKU's

UV bulb	CP1015
---------	--------

Specifications

Dimensions

- 15” wide
- 9.5½” high
- 8.25” deep

Electrical

- 110 - 120 V
- 50 - 60 Hz
- 15 watts
- .5 amps

Warranty Statement

The manufacturer (the “warrantor”) warrants to the original purchaser that this equipment will be free from defects under normal and proper operation for a period of one (1) year from the date of purchase. This warranty covers all parts and labor for that one (1) year period. This warranty shall not apply to any equipment that has been subjected to abuse, misuse, and alteration of any type or to any defect or damage caused by repair, replacement, substitution or use with parts other than parts approved by the manufacturer.

The manufacturer shall not be liable for the loss of use of this equipment, loss of time or business, inconvenience or any other indirect, consequential, special or incidental damages due to defects covered by this warranty or any implied warranty.

To implement this warranty, contact the dealer from whom you purchased the equipment. If necessary, you will be asked to return all, or the appropriate part, of the equipment. If warranted, the repairs will be performed at no charge and the equipment returned to you. Unit owner must prepay all shipping costs when returning items for warranty service.

This limited warranty is the only written or express warranty given by the manufacturer. This warranty gives you specific legal rights but you may also have other rights, which may vary from state to state or province to province.

Authorized Technical Service Center:

Universal Companies, Inc.
18260 Oak Park Drive
Abingdon, VA 24210
Phone: 1-800-558-5571
Fax: 1-800-237-7199
E-Mail: info@universalcompanies.com

Additional Notes

DISTRIBUTED BY UNIVERSAL COMPANIES



Universal Companies, Inc., 18260 Oak Park Drive, Abingdon, VA 24210

Phone: 800.558.5571 **Fax:** 800.237.7199

Web: www.universalcompanies.com

Email: info@universalcompanies.com
