



# PARAGON<sup>®</sup>

## 203 HIGH FREQUENCY

**ATTENTION** Recommended for use by professional, state-licensed estheticians. Operator shall be responsible for and damages to him/herself, the machine, and/or clients.

Before installing and/or using this unit, please carefully read these instructions. Be sure to keep this available for later reference.

### WARNING

**REMOVE ALL JEWELRY PRIOR TO USING THE HIGH FREQUENCY.**

**HIGH FREQUENCY SHOULD NOT BE USED WHEN ON CLIENTS WHO ARE PREGNANT, WHO MAY HAVE A PACE MAKER, OR THOSE WHO ARE EPILEPTIC.**

**NEVER USE ON AREAS MOISTENED WITH INFLAMMABLE LIQUIDS (ALCOHOL, ETHER, ETC.) SPARKS COULD IGNITE THE LIQUIDS CAUSING SEVERE BURNS. THIS TYPE OF LIQUID/LOTION CAN BE APPLIED AFTER ENDING USE OF HIGH FREQUENCY.**

**WARNING: BEFORE PLUGGING IN THE UNIT OR TURNING THE UNIT ON/OFF, BE SURE TO RESET THE VOLUME CONTROL KNOB TO ITS MINIMUM SETTING. STARTING THE UNIT AT ITS MAXIMUM SETTING MAY DAMAGE THE FUSE.**

The High-Frequency machine generates a current characterized by a high rate of oscillation. The current is thermal (or heat producing), and germicidal. This current is also known as Tesla Current.

Included with this unit should be a pre-assembled complete handle and 4 glass electrodes.

### OPERATION

- A. Insert the electrode into the handle.
- B. Turn on the machine by using the green toggle switch.  
TIP: To avoid sparking, operators should first have the electrode in contact with the client's skin before turning on the power switch.
- C. Use the intensity knob to adjust the power according to your client's sensitivity.
  - a. To increase the power of the electrode, turn the knob clockwise.
  - b. To decrease the power of the electrode, turn the intensity knob counter clockwise.
- D. During the process do not let the electrode jump then touch the area being treatment. This may cause sparking.
- E. When changing electrodes, adjust the intensity knob to MIN then turn power switch to the OFF position. Remove the electrode from the holder and insert the desired electrode into the holder. Turn the power switch to the ON position.
- F. After treatment reset the intensity knob to its minimum position. Store in a cool dry place.

### MAINTENANCE

Attachments should be sanitized after each use.

- A. Wipe glass electrodes and attachments with rubbing alcohol and a soft cloth.
- B. Dry electrodes and store in a sanitizer cabinet or autoclave.

### TROUBLESHOOTING

When machine fails to work properly, use the following directions to detect the cause of the problem:

- A. Check the power cord for proper connection.
- B. Make sure the power switch is in the ON position.
- C. Check the power source by connecting another electrical appliance to the AC outlet.
- D. When a glass electrode does not glow, use another electrode to charge the dull electrode. Cross the two like an "X" charging for 1 minute. After this process, both electrodes should glow. If the electrode(s) still do not glow, the electrode(s) may need to be replaced.
- E. If the problem persists, please contact your dealer for assistance.
- F. To prevent an electrical shock, PLEASE DO NOT REMOVE THE COVER.

**ELECTRODES**

Available in Argon-Violet or Neon-Orange.  
Preferences must be stated at the time of purchase.

Larger areas of face and body



Mushroom (203-07)



Spoon (203-09)

Smaller areas of the face and body



Straight (203-11)



Pointer (203-13)

Scalp and Hair



Comb (203-17)



Rake (203-19)

**WARRANTY**

- A. 1-Year warranty against manufacture defect.
- B. Glass parts/attachments not warranty item(s)

**Service**

This is a commercial use appliance, and all servicing should be performed by an authorized representative.

**Obligation**

In order to obtain warranty service, the product must be delivered to and picked up from the manufacturer at the user's expense, unless specifically stated otherwise in this warranty.

**Prolonged Storage**

If the appliance will not be used for a period of time, please empty all water/ lotion/ toners from containers and wrap the cord around the tower stand, store in a cool/dry area.

**Variable Settings            120V, 144W, 0.20amps**

GARFIELD INTL. CO., INC.  
 15977 HERON AVENUE  
 LA MIRADA, CA 90638  
 PARAGON® is a trademark of Garfield International



Questions or Comments?  
 Please call 1-888-731-6688



[www.garfieldint.com](http://www.garfieldint.com)

**ATTENTION**

For your protection in the event of theft or loss of this product, please fill in the information below for you own personal records.

**Model No** \_\_\_\_\_ **Serial No.** \_\_\_\_\_  
(Located on back or bottom side of unit.)

**Date of Purchase** \_\_\_\_\_ **Purchase Price** \_\_\_\_\_

**Purchased From** \_\_\_\_\_