



Introduction to DermaRadiance™ Pure Flower Peel and Diamond Wand Technology

Unique Approach to Treating the Skin

Satin Smooth® DermaRadiance™ Pure Flower Peel respects skin integrity and provides a simple approach to healthy, youthful skin for men and women of all ages. It offers the most sophisticated and advanced systems to regenerate and revitalize the skin. Whatever the aesthetic skin condition, DermaRadiance™ works to regenerate the skin to a healthy state in a gentle, non-invasive manner. This unique system is the first to offer the choice of using Organic Flower Micrograins and/or the Diamond tip wand. Being equipped with two separate and independently functioning wands the technician can offer a much more progressive treatment on widely diverse skin types. Using one or both of the two specially designed hand pieces, the top layer of the stratum corneum (dead cells) is removed to reveal the “fresh” skin just below the surface, creating a glowing new complexion. This unique dual system stimulates sub-dermal activities that “renew” anti-aging mechanisms of the skin that slow with age. Long-term results with DermaRadiance™ are superior because of the substantial difference in our suction technology and its impact upon the skin. The very noticeable appearance of collagen regrowth and healthy, vital skin in people who have received DermaRadiance™ treatments is testimony to the multi-faceted benefits of our two-element approach. Many sources relate this directly to the regenerating stimulation created by the controlled suction of our systems. The dual control feature allows for the separate adjustment of both suction and crystal/Pure grains flow, so that all skin conditions can be rejuvenated. We receive many reports detailing remarkable improvements in the skin, which are further substantiated through objective clinical studies. Because of the controls and capabilities of the DermaRadiance™, thin-skinned areas around the eye and neck can also benefit from this technology. Microdermabrasion removes 10-100 microns of skin. The depth and progression of the treatment is determined by:

- (1) skin condition,**
- (2) micro grain abrasion/suction level settings, and**
- (3) number of passes**

Remember, microdermabrasion is supposed to be a progressive, not an aggressive procedure. DermaRadiance’s Microdermabrasion can mirror a modern method of microdermabrasion. Because of the controlled depth and specialized techniques offered by advanced DermaRadiance™ technology, faster and more dramatic results are achievable in a more even, controlled pattern, with a more progressive regenerative time. It is our less-aggressive, series approach that has now become the staple of skin care programs throughout the world. Microdermabrasion is so universal that both the medical and spa industries are finding ways it complements their business to provide unmatched results for their clients.



DermaRadiance can be effective in reducing the appearance of:

- Hyperpigmentation
- Fine Lines and Wrinkles
- Acne Scars and Active Acne Patterns
- Blending Post- Laser areas
- Sun Damage
- Uneven Skin Tones
- Enlarged Pores
- Age Spots

Advantages of Microdermabrasion over acid methods of exfoliation:

1. Immediate visible results as a result of the vacuum action on the skin stretching the skin and making it tighter; therefore reducing the signs of fine lines and wrinkles.
2. The vacuum action also produces hyperemia, increased blood circulation to the surface of the skin which increases fibroblast activity which in turn causes an increase in collagen and elastin formation over a short period of time, while fruit peels only create inflammation.
3. The removal of dead skin cells allows for more topical serum penetration of nutrients into the dermis. This in turn will facilitate the formation of a new stratum corneum layer to the skin.
4. DermaRadiance does not exhibit any of the acid burn characteristics associated with acid peels and there is no appreciable downtime.

One treatment is equivalent to 3 - 30% consecutive glycolic acid peels or 1 70% glycolic acid peel.

Frequency of treatments

Results will be noticeable from the first treatment, however,

DermaRadiance™ works best as a series of treatments. Follow the guidelines of DermaRadiance™ International's Anti-Aging concept of:

Prevention - Intervention – Rejuvenation

Initial treatments can be spaced a week apart with longer periods of time in between as the program progresses. Consideration should always be given to the professional evaluation of the client's level of skin health in determining frequency and duration of treatments. After the initial series, maintenance treatments are recommended. Frequency is dependent upon need, convenience, and financial considerations. We recommend maintenance treatments every 3 - 6 weeks. People with a tendency to hyper-pigmentation should avoid summer treatments and anyone planning a vacation with a large amount of beach/swimming time should also avoid treatments prior to the trip. All clients should practice diligent use of sun protection especially after a treatment.



Dual controls

The ability to individualize treatments ensures consistent, optimum results. With one control; suction regulator, determining below-surface stimulation and the other; Diamond abrasion tip and/or the quantity of flow of the Organic Micro grains, affecting surface exfoliation, they work together to deliver a complete skin-rejuvenating experience. These dual controls as well as the 2 separate wands have also been designed for convenience for the technician. Its' as easy as flipping a switch to change from one wand to the other.

Duration of treatment

DermaRadiance™ treatments average 15-20 minutes. When preparation and finishing time are included, it is complete within a basic 60 min. treatment. Adding other skin care services can enhance the experience and provide further benefits to the skin program. The neck is included in the standard treatment. By adding other areas (ie. hands, elbows, décolletage etc.) both client-patient satisfaction and facility revenue can increase.

Suction

Suction controls depth of treatment and stimulates blood flow for oxygenation and nourishment in the dermis, which increase fibroblast activity which in turn causes an increase in collagen and elastin formation over a short period of time, while also activating the lymph system for natural cleansing and draining. Suction is essential for full regenerating possibilities. Easily manipulated controls allow for individual treatments of all skin types and areas of skin.

Length of strokes

The length of hand piece strokes is directly associated to suction strength. On average, 1 ½ - 2 inches for broader areas of the face (i.e. forehead and cheeks) is sufficient. Holding the skin taut without stretching is required for all strokes. Use an Up & Out motion for all passes. For additional passes, or skin conditions needing minimal suction (i.e. thin skin showing many capillaries; skin affected by hyper-pigmentation etc.) use shorter strokes and lower suction, concentrating more on surface work.

Type of Stroke: 1. Brush Strokes

Directions of strokes:

1. Up and Out - to cover all areas
2. Left and Right - to cover all areas

"Microdermabrasion is both science and art. Good 'hands' are as much a part of excellent results as are good techniques. With Satin Smooth specially made microdermabrasion hand piece, crystal flow is directed perpendicular to the surface of the skin, hitting at a 90° angle. A 90 degree or perpendicular angle to the skin is the optimum angle for a properly designed handset to produce a smooth and consistant abrasion of micro-grains are delivered by pressure and with force of the vacuum pressure that enhances movement; kinetic energy of the micro-grains and evacuate them from the hand piece. There should be virtually no residual micro- grains on the skin with this hand piece."



What other products are available that could be used with the DermaRadiance machine:

Aluminum oxide- Corundum, Alumina

These sharp crystals are made of aluminum oxide (Al₂O₃) - the second-hardest naturally-occurring material (after diamonds). They are used in the production of emery boards because of their abrasive abilities. In nature, they usually contain impurities which give them their unique color. These are highly regarded as valuable gemstones such as *rubies* and *emeralds*!

Some systems leave a crystal mess all over the treatment room, the operator and patient, too! DermaRadiance, a microdermabrasion systems is efficient, using 50% less product than other systems! Be the first to offer pure *organic* microdermabrasion with DermaRadiance™, the newest, most innovative system in microdermabrasion technology. This unique system is specially engineered to deliver an exciting, new proprietary blend of Pure ingredients for microdermabrasion treatments.

DermaRadiance Pure Flower Grains are:

Safe

Contain No artificial ingredients

Contain No harsh salts

Chemical-free

100% organic

No scratching

Now you can treat the skin more frequently and increase benefits such as stimulation of collagen and elastin production, nourishing blood stimulation, and oxygenation to the skin. Resulting in fresh, renewed and smooth skin.

Warnings / Contraindications

Full Treatment Chemical Peel

(i.e. Jessner, Phenol, Obagi) Although we suggest a series of DermaRadiance™ treatments instead, it is best to complete any peel program and allow 7 days for full healing. Then use DermaRadiance™ treatments as maintenance unless prescribed alternatively by a physician.

Collagen Injection

If planning a treatment using collagen injections, first complete the *DermaRadiance*™ series of treatments, and then the injections. Wait 5-6 weeks before your next DermaRadiance™ treatment.

Topical Collagen Application

First complete DermaRadiance™ treatment, followed by collagen.



Botox

First, a *DermaRadiance*™ treatment, then wait approximately 5 days before Botox injections. Wait for a two week period before next microdermabrasion treatment.

Electrolysis

Skin sensitivity is a key factor in combining electrolysis with microdermabrasion. Perform the hair removal treatment first. If the client has minimal discomfort and swelling, wait two to three days before the *DermaRadiance*™ treatment. If the client is more sensitive, wait at least a week.

Waxing

Some waxes are specific to only removing hair while others remove skin layers. Either way the sensitivity of the skin makes combining these treatments on the same day very uncomfortable. We suggest 5-7 days between treatments.

Photofacial

DermaRadiance™ treatment first, then wait a week before *Light Regeneration (photofacial)* treatment.

Other Accute Contraindications:

Raised moles, warts

Skin cancer

Herpes simplex - Zovirax™ or Acyclovir should be taken 24-48hrs before and after treatment. Avoid treating the lip line; or consult a physician for the best course of action.

Active, uncontrolled diabetes - there is an inherent risk that healing may be impaired.

Keloid scars - are to be approached through medical treatments only.

Telangiectasia - should be corrected before treatment, however other areas of the face can be treated.

Roaccutane - should not have been used for at least three months before treatment.

Cautionary Indications

Active acne

Rosacea

Capillaries

If the condition is not severe, use minimal suction and only a few light passes on fine, thin areas of the skin showing visible capillaries. Avoid using suction on areas where many capillaries are clearly visible through the skin.

Pregnancy

Although there has never been any problem reported, we recommend that you obtain approval from the client's attending physician.



Hyper-pigmentation

Use fast, short strokes with high crystal volume over areas showing dark, pigmented areas. Look to deliver an even color to the complexion during initial treatments and then concentrate on eliminating dark areas and spots in subsequent treatments. Exercise patience in treating this condition because over-aggravation of melanocytes will recreate the condition. Home treatment program requires using Satin Smooth® Oxygen Infusion Serum and Collagen Complex Cream.

Altered pigmentation/hyperpigmentation

Melanocytes are located in the *stratum germinativum* of the epidermis. When stimulated by heat, or the uv rays of the sun, they release melanin into the skin, causing the darker pigment on the surface. With our dual-control systems and advanced *positive pressure* systems with their emphasis on crystal projection, we can focus and concentrate our work on the surface with little stimulation of the melanocytes. Supplemental DermaRadiance™ and home-care is essential for controlling this condition.

Blackheads/ Comedones

The DermaRadiance™ System is very effective in loosening blackheads. Many blackheads are removed during the treatments, and more release over the next couple of days. In some cases after the DermaRadiance™ treatments, the skin will have a light, manageable peeling in certain areas. Using a light granular scrub will help to minimize flaking and continue to loosen trapped debris (i.e. blackheads and whiteheads). Application of Satin Smooth's Oxygen Infusion Serum will control peeling.

Temperamental Breakouts

The cleansing and balancing effect of the suction combined with the DermaRadiance™ surface refinement is an incredible new way to control and minimize skin breakouts. Trapped debris is released, thereby limiting pore infection. DermaRadiance is also effective in disrupting acne patterns. Caution should be used when working around open lesions.

Permanent Make-Up

Wait one month after application before the first DermaRadiance™ Treatment. If considering permanent make-up, or re-treating an area, a DermaRadiance™ treatment first will help to improve application results.

After Treatment Make-Up

If possible, avoid applying make-up for at least 24 hours after the treatment. The pores have been opened and it is an excellent time to apply serums and nutrients. Allow the pores to close naturally before applying make-up. If circumstances do not allow for this time, use of a barrier (day) cream is essential to prevent absorption of make-up into the dermis. Even after the first treatment, skin is so smooth that makeup can be applied more easily and quickly - and less is used. *Please note; sun protection should ALWAYS be applied subsequent to a DermaRadiance treatment.



Men's Shaving

Men additionally benefit by reducing shaving time and effort, assisting with shavers' rash and ingrown hairs, and decreasing grooming time.

What to Expect Right After Treatment

All skin respond differently. Some skins are pink, some are reddish in spots, and some just show a healthy, even color. Most color, which is erythema—not inflammation—subsides within the first four hours, although some areas can remain pink for 24-48 hours. This can be in a spot area, most likely cheeks, or as a mark that looks like a streak. This is more common in clients who smoke or have very thin/sensitive skin. It will subside. There is definite heat in the skin which often increases at night. Avoid strenuous exercise that evening to prevent excess blood stimulation to the face. A prickly sensation may be experienced at the time of perspiration - this is not an allergic reaction. Microdermabrasion is comparable to a 40-70% Fruit Acid or Glycolic peel

Frequently Asked Questions:

1) How taut the skin is when held:

The action provided by the suction-pressure directly affects the depth of abrasion and the intensity of the surface crystal/micro-grain exfoliation. If you envision the tautness of a drum head, when loose there is more absorption; when taut, there is more impact. The same is true with the skin. The suction action also delivers nourishment and oxygenation from stimulated blood circulation, and an effective lymphatic drainage cleansing, which helps reduce puffiness. We have found suction to be essential for the full regenerating benefits of microdermabrasion.

2) The volume of crystals/ micro-grains:

The amount of crystals/micro-grains that are projected onto the skin determines the amount of exfoliation.

3) Speed of movement:

Speed is directly connected to depth of abrasion, since faster strokes are less stimulating ie. when treating sensitive skin; faster strokes should be used.

Preparation

Firstly, we recommend that you observe traditional precautions in preparing for any treatment involving client/patient contact by wearing gloves and a face mask. Most clients are used to these safety measures, and it will reinforce your own professional standards.



Next, position the client in a reclining manner. Cover the hair with a turban or cap. Remove jewelery. Removing contacts is optional, since the micro-grain residue from DermaRadiance™ is minimal. Follow cleansing steps. To commence treatment, secure the skin with the thumb and middle finger, holding it taut - not stretched. Place the handpiece nozzle into position on the skin, beginning right below the jaw-line. Hold the hand piece upright, at a 90 angle, as if it were a pen and engage suction, beginning the treatment. When first beginning the treatment, begin slowly, allowing the skin's reactions to match your confidence level.

Sensitivity Test

Before beginning, let the client experience the sensation of the vacuum suction and micro-grains projection by testing the treatment on the client's cleansed, dry forearm, so he/she knows what to expect.

Approach to Sensitive Areas

Start gently at the corner of the eye (temple), using a slow brush stroke/sweeping action, up and out toward the hairline. Continue with slightly overlapping strokes. Then, complete the other side of the face. On most areas of the face stroke length should be 2-3 inches, when applicable. This maintains suction connection. **Do not exert any pressure on the skin; the strokes must be light. Allow the vacuum action of the hand piece to engage the skin, guide the hand piece up and outwards.**

Use an up and out stroking motion - this stimulates the lymphatic system, which helps cleanse and reduce fluids, (i.e. puffiness). When working on the cheek bone area, proceed gently, as this area is rather sensitive. Remember that microdermabrasion works on three principles: ***Prevention, Intervention and Regeneration.***



Contraindications:

Raised moles, warts

Skin cancer

Herpes simplex - *Zovirax™ or Acyclovir should be taken 24-48hrs before and after treatment. Avoid treating the lip line; or consult a physician for the best course of action.*

Active, uncontrolled diabetes - there is an inherent risk that healing may be impaired.

Keloid scars - are to be approached through medical treatments only.

Telangiectasia - should be corrected before treatment, however other areas of the face can be treated.

Roaccutane - should not have been used for at least three months before treatment.

Cautionary Indications

Active acne

Rosacea

Capillaries

If the condition is not severe, use minimal suction and only a few light passes on fine, thin areas of the skin showing visible capillaries. Avoid using suction on areas where many capillaries are clearly visible through the skin.

Pregnancy

Although there has never been any problem reported, we recommend that you obtain approval from the client's attending physician.

Hyper-pigmentation

Use fast, short strokes with high crystal volume over areas showing dark, pigmented areas. Look to deliver an *even color* to the complexion during initial treatments and then concentrate on eliminating dark areas and spots in subsequent treatments. Exercise patience in treating this condition because over-aggravation of melanocytes will recreate the condition. Home treatment program requires using *Satin Smooth® Oxygen Infusion Serum and Collagen Complex Cream*.

More Treatment Tips

DermaRadiance™ treatments are effective for many areas of the face and body. Try microdermabrasion on elbows, knees, hands, feet, back, chest, and décolleté. It is also effective on *follicular keratitis* - the bumpy skin often on the backs of the upper arms.

-Don't forget lips, eyebrows, and behind the ears.

-Advise all clients that a "fresh skin needs a fresh pillow case".

-If possible wait 24 hours before applying makeup. If not, wait as long as possible and always apply a barrier/day cream first, so makeup is not drawn into fresh, open pores.

Hydrate, hydrate, hydrate! Especially during the first 3-4 days after treatment. We have affected the water level of the skin and its ability to hold moisture. We recommend liberal use of Oxygen Infusion Serum.



Sun Protection. Essential in all microdermabrasion treatment programs is the use of sun protection on a daily basis. Please ensure sun protection after all day time treatments.

Examining the Skin Before Treatment

Identify basic skin types. Use the *Client Information Card* to determine skin condition and previous treatment history. Use first series of passes as an indication of client sensitivity, with regard to the number of passes to be used in the Second Pass. Customization of treatment can be used by combining miro-grain treatment and Diamond wand.

Sensitive areas such as eyes and brows need special attention. Press your thumb between the eyebrows. If, when lifted, a white mark remains, be aware that the skin is sensitive. Look for areas of visible capillary networks. **Use caution in this area** - the duration of treatments should be lowered and minimum suction used. Also use a lower suction pressure where areas of hyper-pigmentation have been identified. Treat active acne according to treatment instructions.

Wood's Light

Alternative assessments can be made using ultraviolet photography or Wood's light - an inspection lamp that produces deep ultraviolet rays. Using such tools allows different types of skin to characteristically fluoresce or glow in different colors and hues to expose skin conditions that may not appear obvious to the naked eye.

Did you know? DermaRadiance™ systems allow you to vary treatment depth and duration according to each individual's skin. You can always go back and repeat passes over specific areas after waiting a few minutes and evaluating the skin's response. Be conservative when treating new clients.

Note: The DermaRadiance™ Pure Flower Peel system gives you the ability to control the treatment, adjusting levels of pressure and crystal/micro-grains flow for different parts of the face.

Customizing the DermaRadiance™ Pure Flower Peel Treatment to each individual assures great results and a high level of recipient satisfaction.

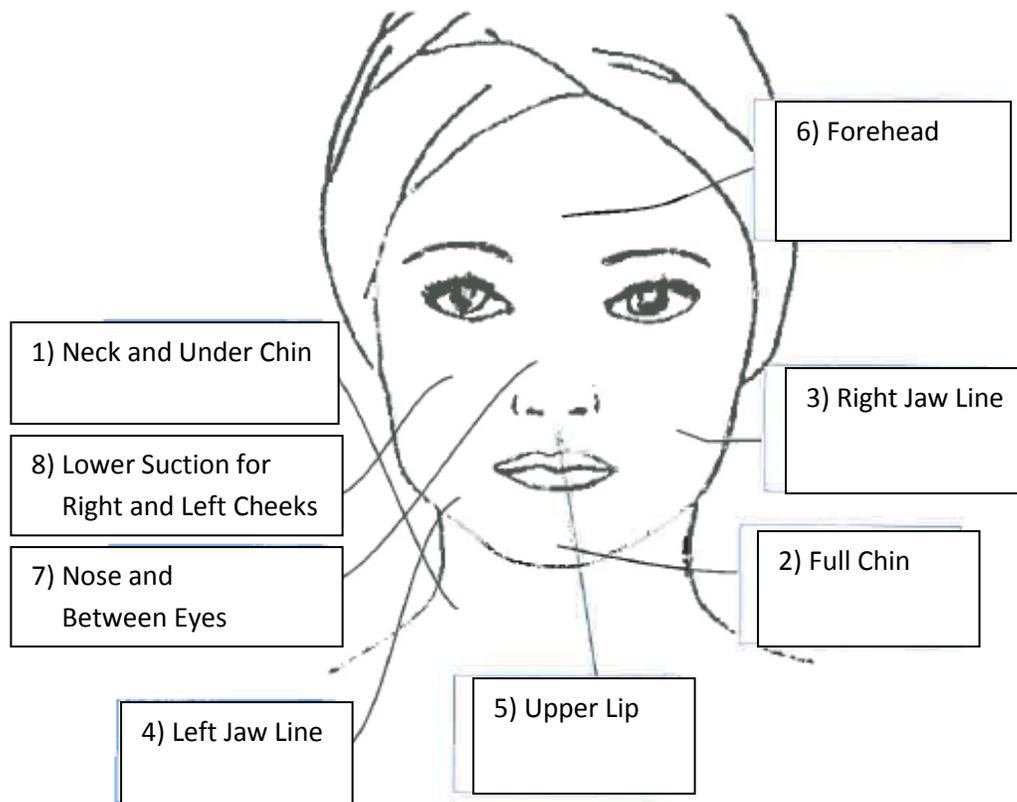


Image 1 – Facial Zones for DermaRadiance Treatments

10 minutes with Diamond Wand, followed by 10 minutes of Flower Grains will leave you clients skin Radiant.

Using the Crystal - Pure Flower Grains

Step One: Cleansing Action

Cleanse the face and neck with Satin Smooth Facial Cleansing Milk to lift impurities from the skin without drying or leaving a film. Massage with finger tips and remove using warm water and towel.

Step Two: Preparing the skin

Remove excess moisture, leaving the skin completely dry - do not use cotton wool or tissues. Use a dry Kleenex tissue.



Step Three: Select Pure Grains best suited for their skin type

Satin Smooth Pure Lavender Flower Grains: To soothe and hydrate stressed skin.

Satin Smooth Pure Rose Flower Grains: To balance and tone skin complexion.

Satin Smooth Pure Tea Tree Flower Grains: Best suited for oily and blemish prone skin.

Step Four: Begin Microdermabrasion Treatment

Begin by placing the Media on level 3 and Suction on level 3. Then adjust depending on clients comfort and skin sensitivity. Lower level if client is uncomfortable higher if skin is not reacting and client is comfortable.

Complete treatment should be performed within 15-20 minutes with 2 full passes using a cross-hatch technique. First motion pass is a vertical and second in a horizontal direction; this is called cross hatching. Following a pattern on the face move from one area to the next using the cross hatching method (See Image 1 above).

Begin with:

- 1) Neck
- 2) Full Chin jaw line
- 3) Right Jaw Line
- 4) Left Jaw Line
- 5) Upper Lip
- 6) Forehead
- 7) Nose and Between Eyes
- 8) Right and Left Cheeks (lower suction)

Each area should have 2 passes (vertical and horizontal) of the cross hatching before moving on to the next area. For sensitive skin; you may want to do 1 complete pass in a vertical direction. After completing the whole neck, jaw line and face, make the second pass in a horizontal direction. This will allow for closer monitoring of the skin.

Recommendation: 10 minutes with Diamond Wand, followed by 10 minutes with Pure Flower Grains or via versa. This produces a polish to the skin.

Step Five: Removing Pure Grains

Remove excess Flower Grains *by using a soft dry cloth or a soft face brush.*

Step Six: Cleanse clients' facial or treated area thoroughly with warm water and light amount of Satin Smooth Facial Cleansing Milk, rinse then pat dry.

Step Seven: Complete service by applying Satin Smooth Oxygen Infusion Serum massage thoroughly for 5 minutes, do not rinse.

Using the Non-Crystal - Diamond Wand

Step One: Cleansing Action

Cleanse the face and neck with Satin Smooth Facial Cleansing Milk to lift impurities from the skin without drying or leaving a film. Massage with finger tips and remove using warm water and a towel. Rinse well.

Step Two: Preparing the skin

Remove excess moisture, leaving the skin completely dry - do not use cotton wool or tissues. Use Kleenex tissues.

Step Three: Set Up

Settings: assessing clients' skin and sensitivity select grade of tip for your treatment. (Grade 1 to 5). Use tips cautiously. Satin Smooth recommends using *tips 1, 2 & 3 for facial exfoliation*. *Tips 4 and 5 are only used on body areas, such as the elbows and hands.*



Attach a Diamond Tip by screwing the attachment into Diamond Wand. Place Diamond Wand adjustable head over the diamond tip, using one of the four adjustment grooves on side of hand piece. This adjustment will provide light – deeper penetration of vacuum to the skin.






Handpiece Control Notch



Diamond Wand Handpiece

**Diamond Adjustable
Head Grooves
(Positions 1-4)**

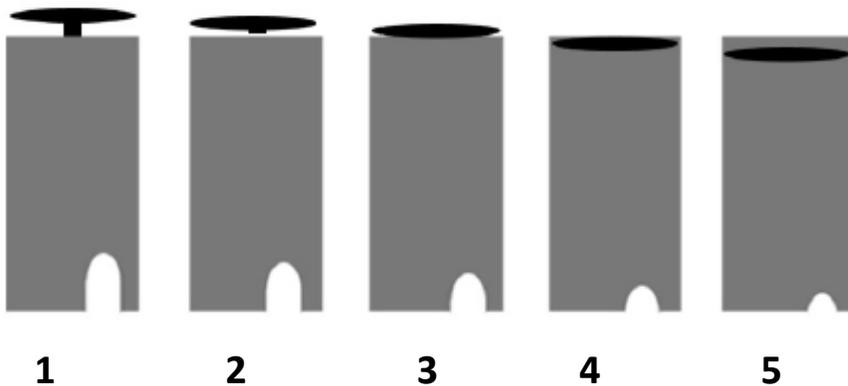

Suction Cap Settling 1 – 4

- 1) Suction = maximum – very high
Peel = minimum – very low
- 2) Suction = medium high
Peel = medium low
- 3) Suction = medium
Peel = medium
- 4) Suction = low
Peel = medium high
- 5) Suction = very low
Peel = very high

IMPORTANT: Attach a new blue disposable filter to the Diamond Wand hand piece then attach the black tubing into the disposable filter. A new blue disposable filter should be replaced after 3-6 uses. Changing filter will aid in the maintenance and performance of your machine.





IMPORTANT: Attach a Hepafilter to the Diamond Wand hand piece then attach the other end to machine. A new Hepafilter should be replaced after 6 months. Changing filter will aid in the maintenance and performance of your machine.



Step Four: Beginning Diamond Wand Microdermabrasion

This treatment can be conducted wet or dry. For a wet treatment use Satin Smooth Oxygen Infusion Serum. The diamond wand will help infuse this formula into the skin.

Begin by selecting the Tip 1 for a client's first treatment this will provide a leveled exfoliation and ensure a safe treatment for clients first time experiences. Adjust Suction on level 3. Then adjust depending on clients comfort and skin sensitivity. Lower level if client is uncomfortable higher if skin is not reacting and client is comfortable.

Complete treatment should be preformed within 15-20 minutes with 2 full passes using a cross-hatch technique. First motion pass is a vertical and second in a horizontal direction; this is called cross hatching. Following a pattern on the face move from one area to the next using the cross hatching method (See Image 1 above).

Begin with:

- 1) Neck
- 2) Full Chin
- 3) Right Jaw Line
- 4) Left Jaw Line
- 5) Upper Lip
- 6) Forehead
- 7) Nose and Between Eyes
- 8) Right and Left Cheeks (lower suction)



Each area should have 2 passes (vertical and horizontal) of the cross hatching before moving on to the next area.

Recommendation: 10 minutes with Diamond Wand, followed by 10 minutes with Pure Grains., or via versa, depending on client's skin sensitivity. 2 wands and full passes can be done to achieve maximum exfoliation.

Step Five: Cleanse clients' facial or treated area thoroughly with warm water and light amount of Satin Smooth Facial Cleansing Milk, rinse then pat dry.

Step Six: Complete service by applying Satin Smooth Oxygen Infusion Serum massage thoroughly for 5 minutes, do not rinse.

Microdermabrasion: Treatment for Face & Neck

This exciting new skin therapy quickly rejuvenates your skin and works to refine complexion lines, wrinkles, and diminish brown spots. The result is vibrant, healthy skin with an unmistakable youthful glow. There is no discomfort, down time, lifestyle inconvenience nor chemicals. Optimum results are achieved in a series of treatments.

DermaRadiance™: Face & Neck with Mask

The assimilation quality of the skin after a DermaRadiance™ Pure Flower Peel is exceptional. This is an excellent time to nourish, re-mineralize and re-balance the skin with masks and serums. Treat yourself to this complete rejuvenating program and see how microdermabrasion restores the tone and strength of the skin.

Menu suggestions: After a DermaRadiance exfoliation an add-on treatment of a soothing Satin Smooth Ultimate™ Collagen Mask with completely refreshes, cool and tone the skin.



DermaRadiance™ Patient Information Card (page 1)

Prepared especially for clients considering DermaRadiance™ Pure Flower Peel Treatments. Satin Smooth's MicroDermabrasion is the new, non-invasive, convenient skin care technology that is specially designed for men and women who want to look their best, regardless of age. Satin Smooth does not assume any responsibility to this consent form.

Name: _____

Address: _____

City: _____ State: _____

Occupation: _____ Age: _____

Referred by: _____ Date of consultation: _____

Which conditions would you like to improve?

Hyperpigmentation Sun damage, Acne scarring Age spots



Fine lines & wrinkles Acne no. months _____ years _____
Surgical/facial scars, Enlarged pores, Stretch marks Other (please specify)

Areas to be treated: (Please detail entry here)

Client questions: How sensitive is your skin?

Are you currently under a physician's care? _____

Reason: _____

Are you pregnant or planning to be? _____

Do you have a heart condition? _____ Do you wear a pacemaker? _____

Do you have or have you ever had
acne? _____

What allergies do you have (if
any)? _____

Please list any medication you are on

Do you wear contact lenses? _____

Do you smoke? _____ If not, were you ever a smoker? _____ How many years? _____

Have you visited a dermatologist or other skin care therapist? _____

Reason: _____

What products do you use on your skin? _____

Do you have epilepsy or diabetes? _____

(If yes, this can only be treated with a doctor's certificate).

Do you have or have you ever had any of the following:

Keloid scarring Hepatitis Dermatitis Acne scarring
HIV Eczema Skin cancer Herpes simplex
Other (please specify)

Have you ever had any of the following treatments:

Chemical peel Laser peel Other, similar treatments
Glycolic peel Botox
Cosmetic surgery Other (please specify)



What product are you presently using?

Cleanser Moisturizer

Do you use or have you ever used any of the following products?

Retin A Hydroquinone Roacutane

Other (please specify) Any reactions?

Comments: _____

Please print name clearly:

I _____, do fully understand all the questions above and have answered them all correctly and honestly. Furthermore, I know that it is my responsibility to alert the clinician about any recent surgeries or skin resurfacing procedures. Without the above disclosure I understand that the attending technician cannot optimize the effectiveness of DermaRadiance™ Pure Flower Peel treatments, which are designed to provide clients with superior results. By signing below I consent to the Satin Smooth DermaRadiance™ Pure Flower Peel Microdermabrasion procedure.

Signed by Patient Date

Signed by Technician Date

Note: Please complete both (2) pages. Thank you!